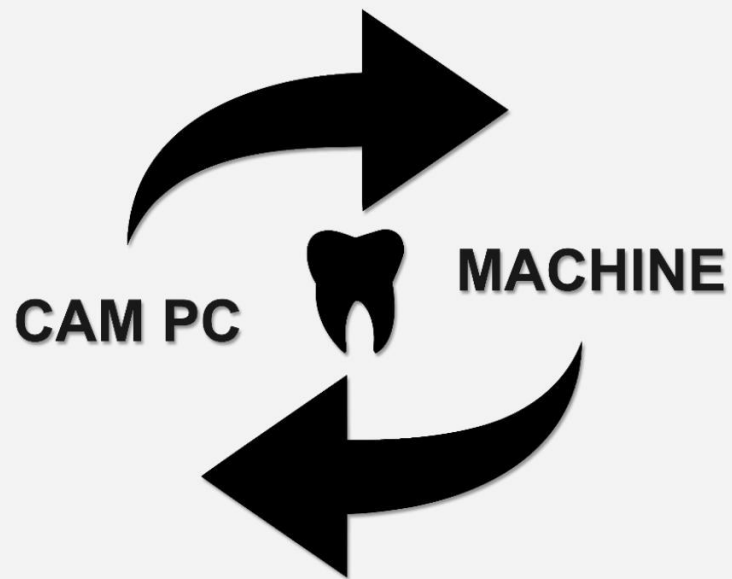
Brief Instruction | Kurzanleitung

CORiTEC 150i

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Information | Informationen

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The contents of this Brief instruction are intended exclusively for trading partners of imes-icore GmbH. A spreading of the guidance to third is forbidden!

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The following prerequisites must be ensured in order to carry out the contents of the Brief Instruction:

- ☐ The acting person must have received a dealer training by imes-icore GmbH!
 - ☐ Use suitable personal protective equipment!
 - ☐ All contents of the included documents (operating manuals, accessories, etc.) must be read and understood!
-

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Copyright | Urheberschutz

The contents of this Brief instruction are copyright protected and are the intellectual property of imes-icore GmbH. Usage of this content is only permissible of using the machine. Any use outside of this is prohibited without the express written permission of the manufacturer. All rights, also those of translation, are reserved.

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Trademark protection | Markenschutz

All rights to product, company and brand names or third-party rights in this operating manual belong to the respective company or holder, regardless of their form, and are subject to international copyright and trademark laws. Individual labelling has been omitted from this Brief instruction. imes-icore® is a legally protected trademark according to section 4 no.1 MarkenG (trademark law).

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Safety | Sicherheit

Personal requirements | Personalanforderungen

The acting person must have received a dealer training by imes-icore GmbH!

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
Protective work clothing, protective gloves, safety footwear, respiratory protection, safety goggles.

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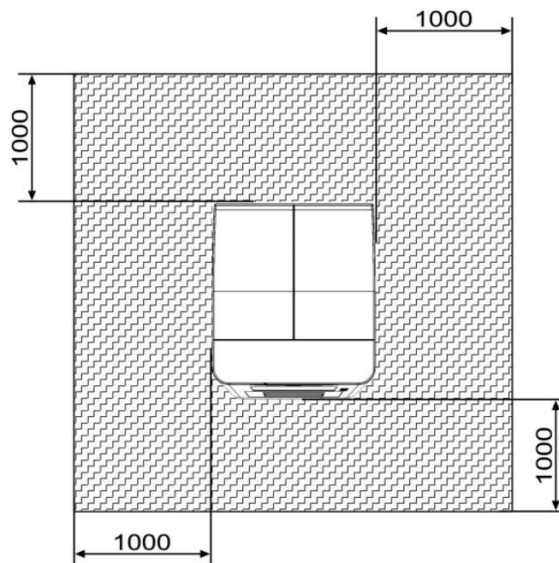Safety instructions | Sicherheitshinweise

⚠ DANGER

Danger to life due to improper commissioning, handling, repair or maintenance!

Improper commissioning, handling, repair or maintenance will result in death, life-threatening injuries or serious damage to the machine!

- ▮ Work on or with the machine must be carried out by qualified and trained personnel!
- ▮ Work on or with the machine must always be carried out strictly in accordance with the manufacturer's documentation!
- ▮ Only use original spare parts from the machine manufacturer
- ▮ Always wear suitable personal protective equipment according to the work to be carried out!

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⚠ DANGER

Danger to life due to electric shock!

The machine has mains voltage! Touching live parts will result in death or serious injury due to electric shock!

- ▮ All maintenance and repair work may only be carried out by qualified electricians!
- ▮ All maintenance and repair work may only be carried out in currentless state of the machine!
- ▮ Remove the power cord and wait a few minutes before you start work!

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⚠ WARNING

Risk of injury from rotating tools and moving axles!

Contact with moving parts of the machine can cause serious injury!

- ▮ Always be careful and follow the general safety rules!
- ▮ Never reach into moving machinery!
- ▮ Wear protection gloves!

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⚠ CAUTION

Risk of injury by swarf!

Swarf in the machine may have sharp edges and cause deep lacerations!

- ▮ Clean the machine thoroughly, always before starting any work!
- ▮ Wear protection gloves and safety goggles!

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
Transport, installation and commissioning of the machine according to the operating manual! Observe the warning and safety instructions in the operating manual!

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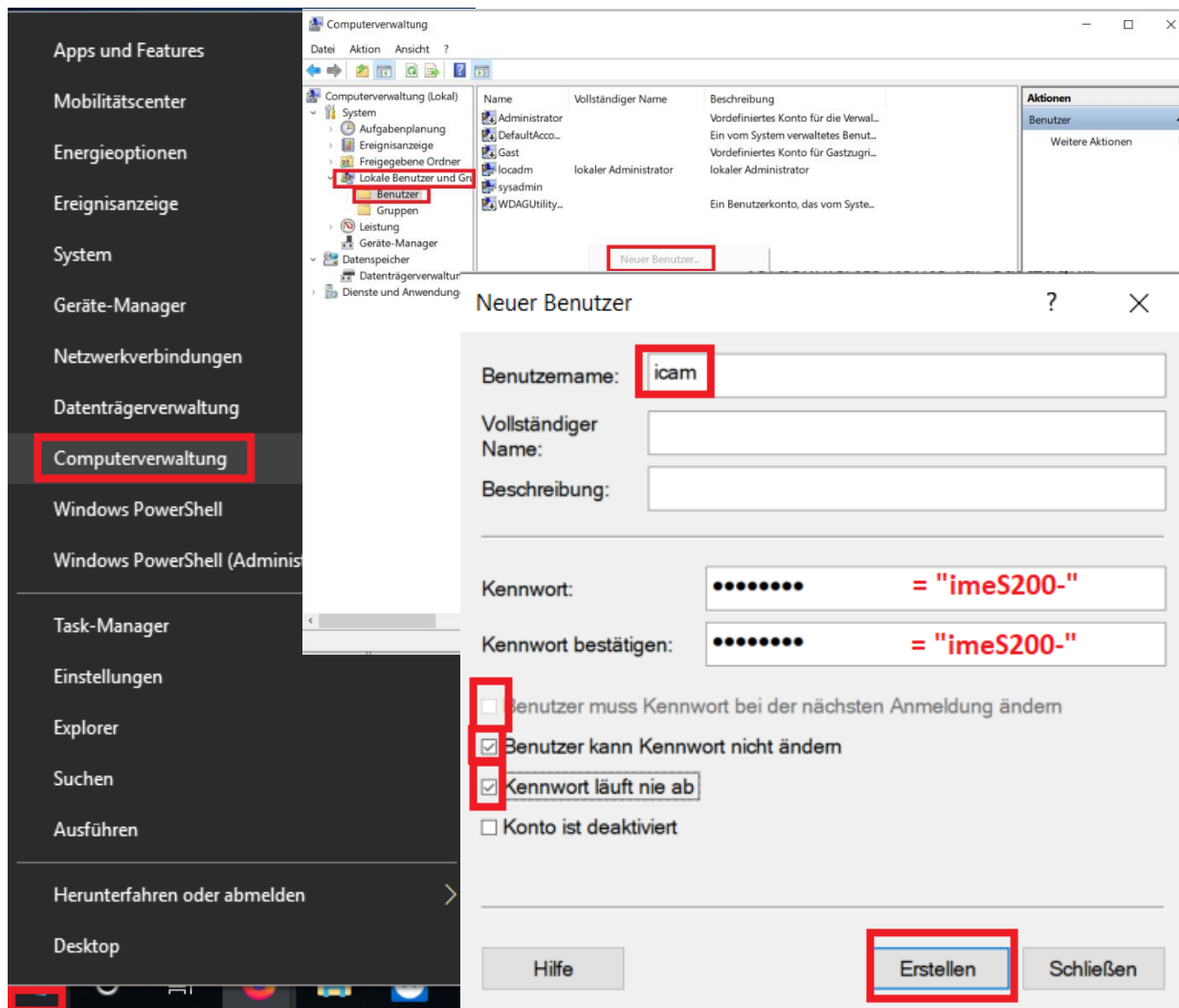1.

Right click to the Windows Symbol
→Computer Management
→Local Users and groups
→Users


Right click to New User..
Then take in all things like in the picture
(also look for the ticks)

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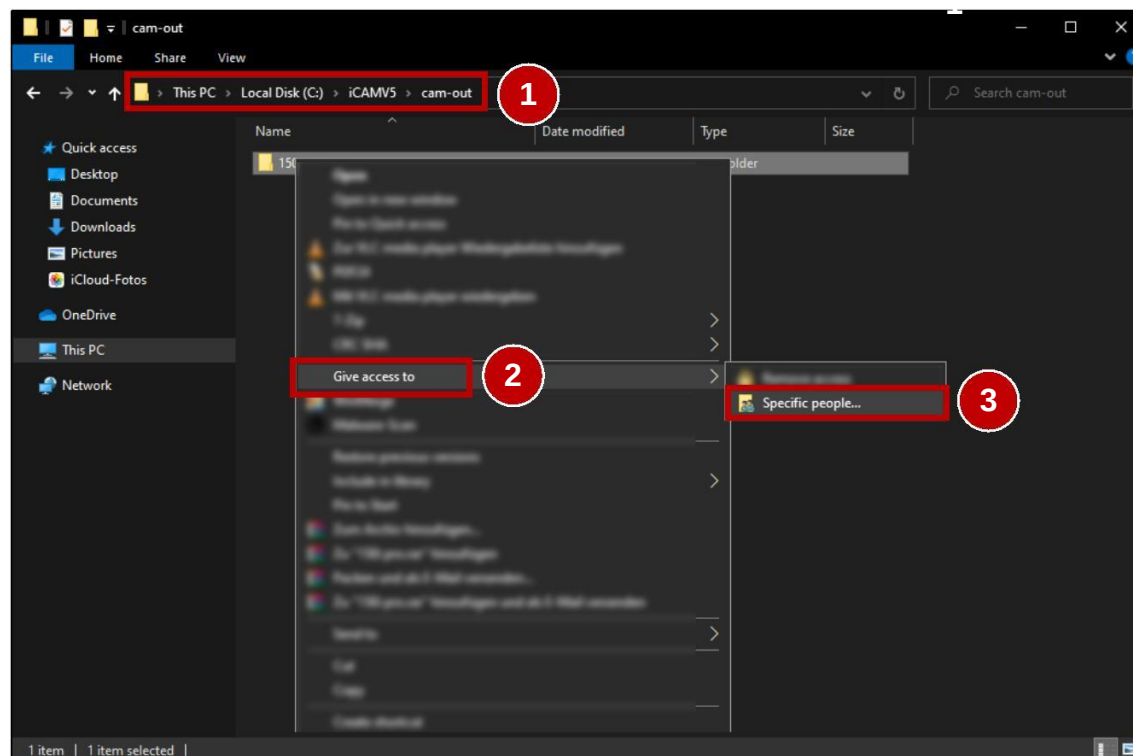
1.

1. Open a new Windows® Explorer window.
(Windows® button + E)
2. Navigate (2) to the CAM output folder.
3. Right-click on the folder to be shared in the network.
4. Open Give access to (2).
5. Press Specific people (3).

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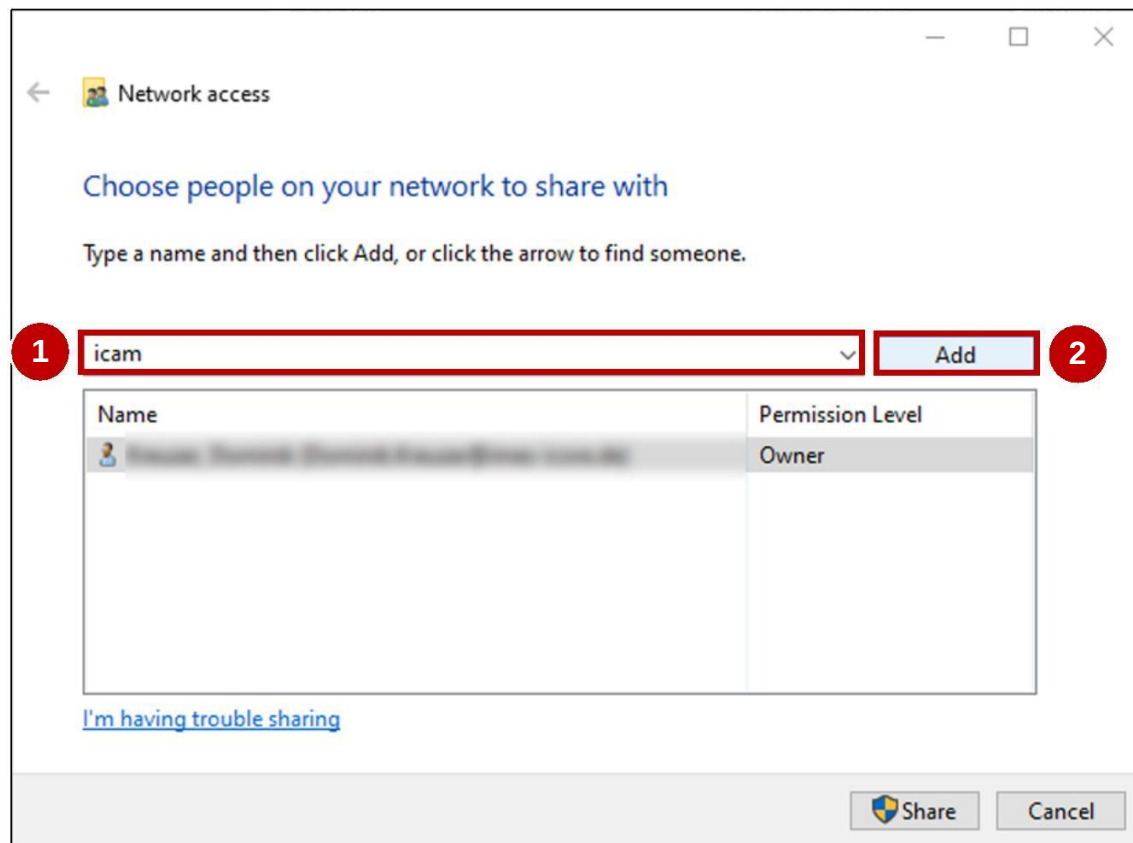
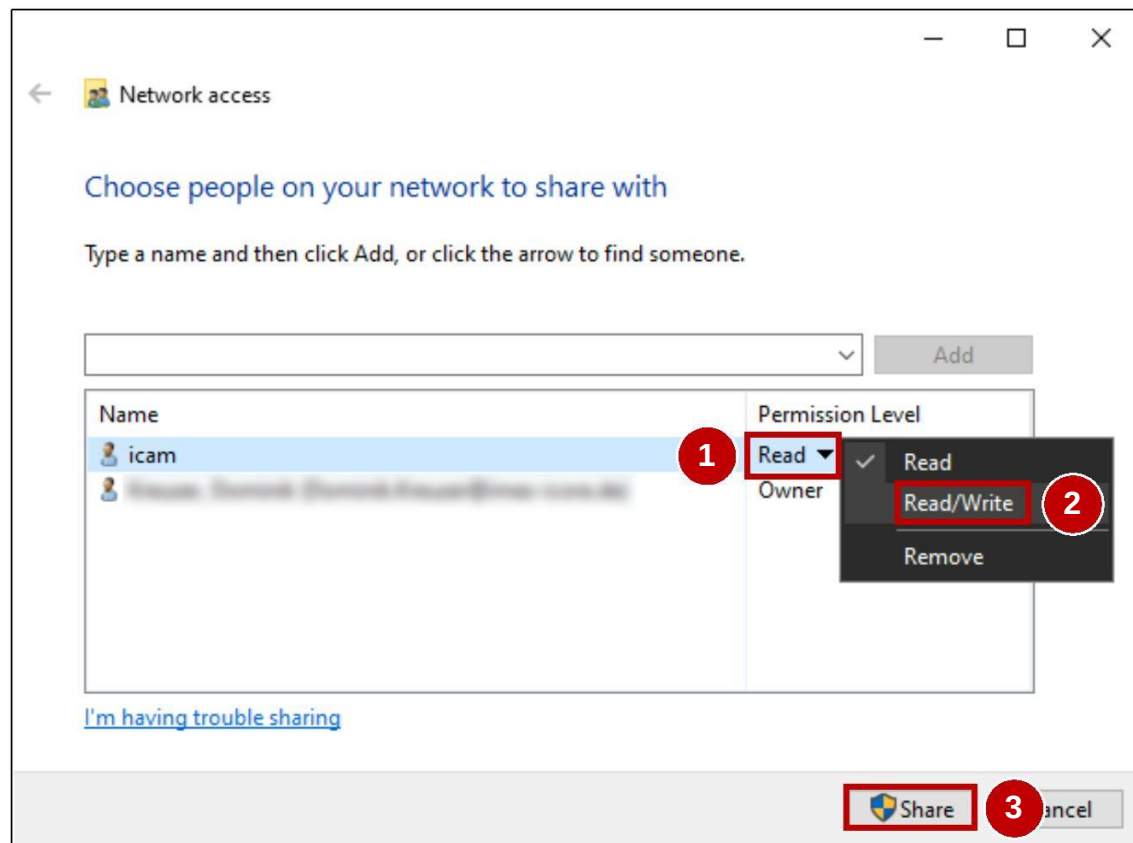
3.

1. Press onto the Permission Level (1) of the User icam.
2. Choose Read/Write (2).
3. Press Share (3).

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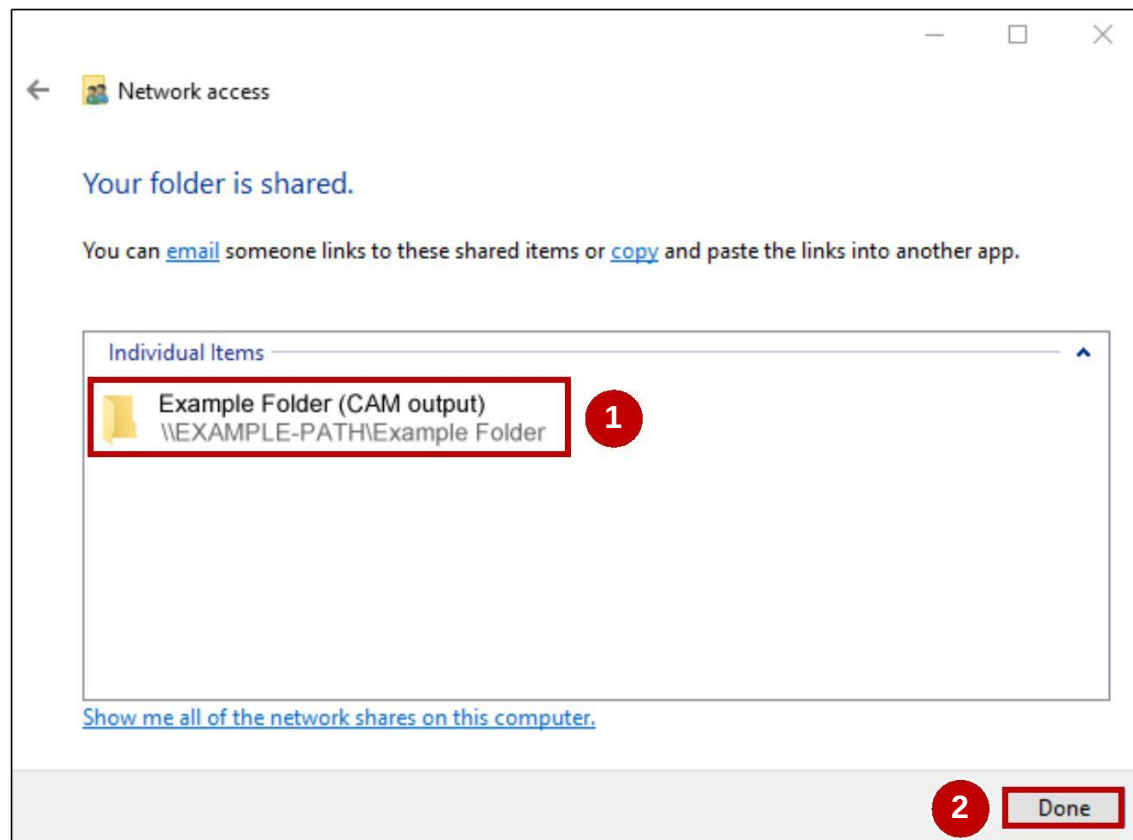4.


1. Check the correctness of the shared folder (1). And take this grey path into the machine.*
2. Press Done (2).
3. The network set-up of the CAM PC is accomplished!

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CORiTEC system with SmartControl V2.0 | CORiTEC System mit SmartControl V2.0

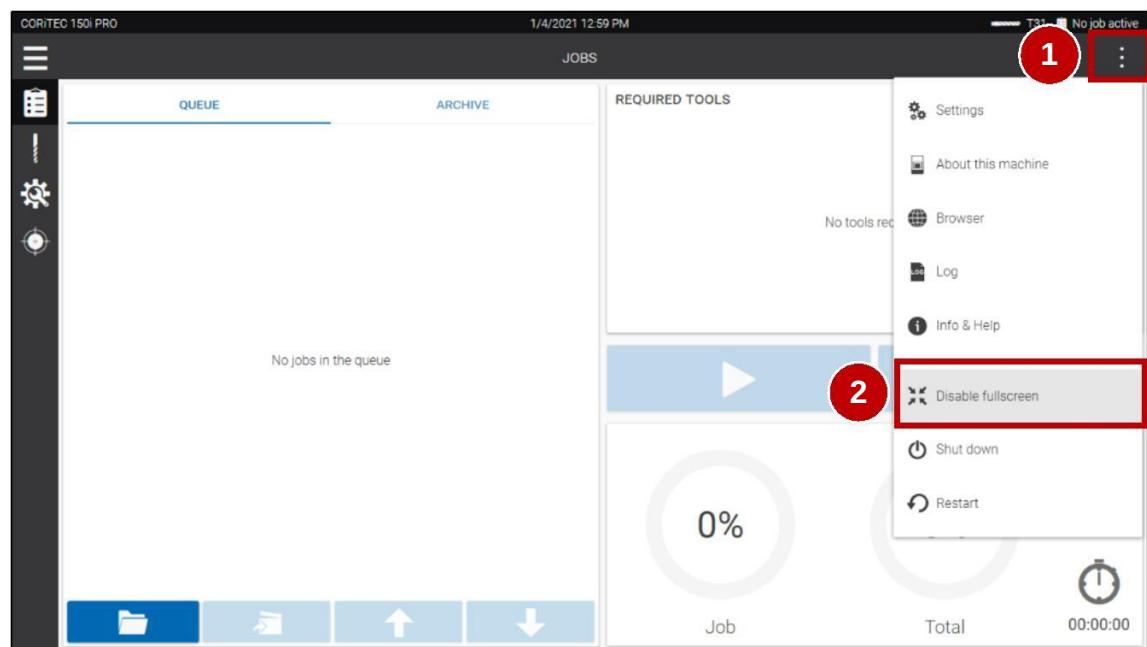
1.

1. Switch on the machine at the main switch.
2. Wait until the Customer UI has completely booted.
3. Press the menu button (1).
4. Press Disable fullscreen (2).

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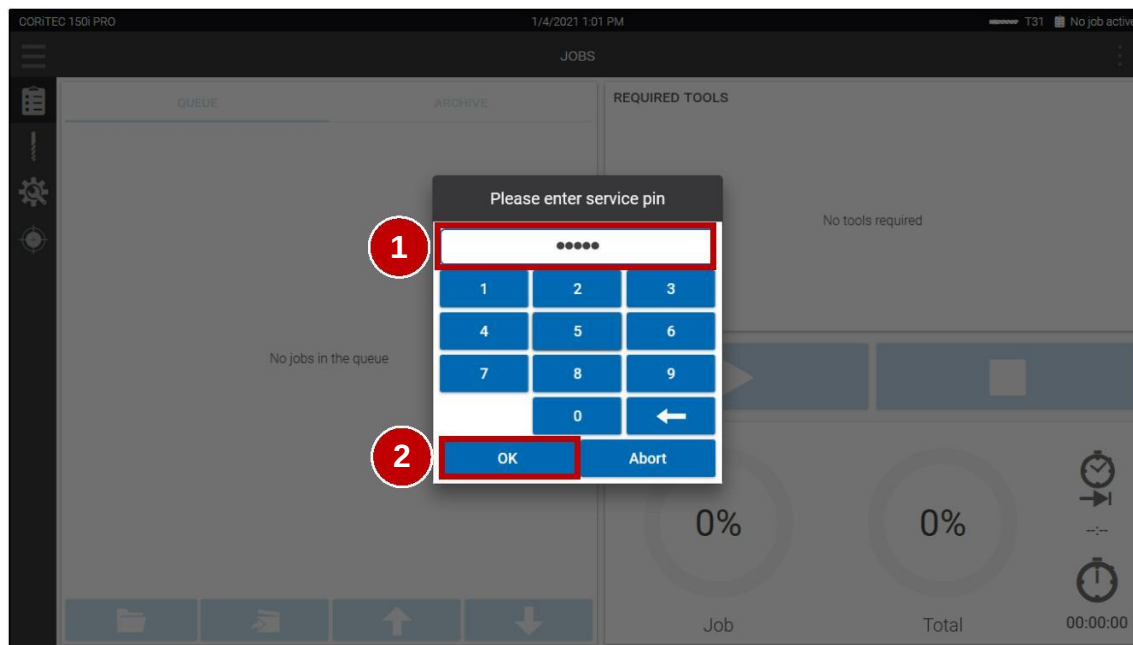
2.

1. Enter Password (1).
(Note: To receive Password contact imes-icore customer service!)
2. Press OK (2).

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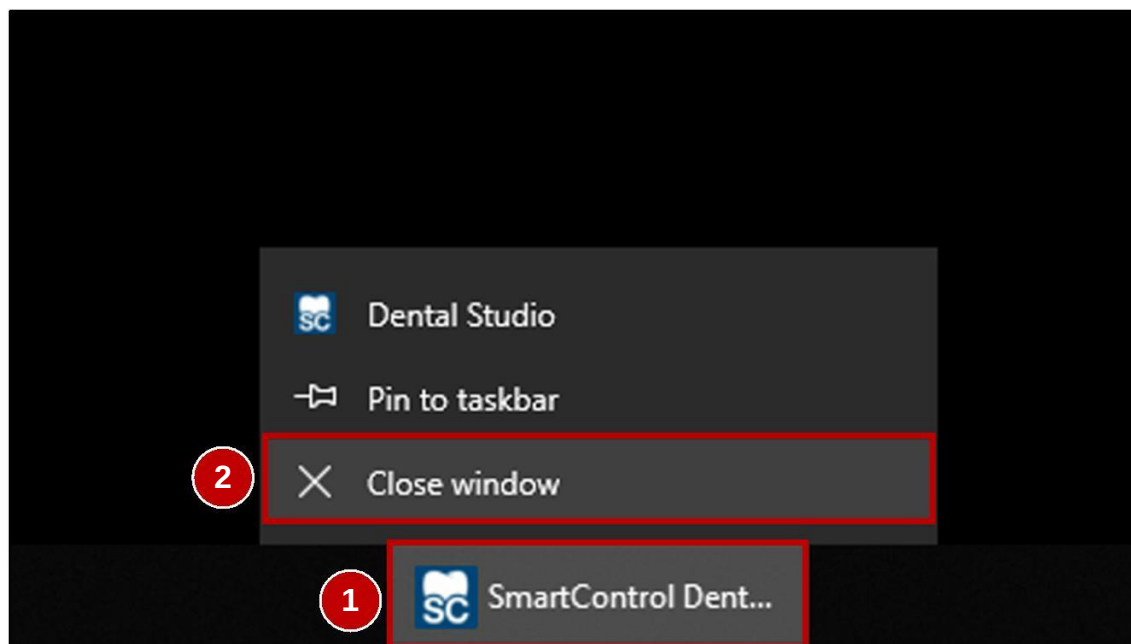
3.

1. Press on and hold on the minimized SmartControl task (1) in the Taskbar to open the context menu.
2. Press Close window (2) to quit the Customer UI.

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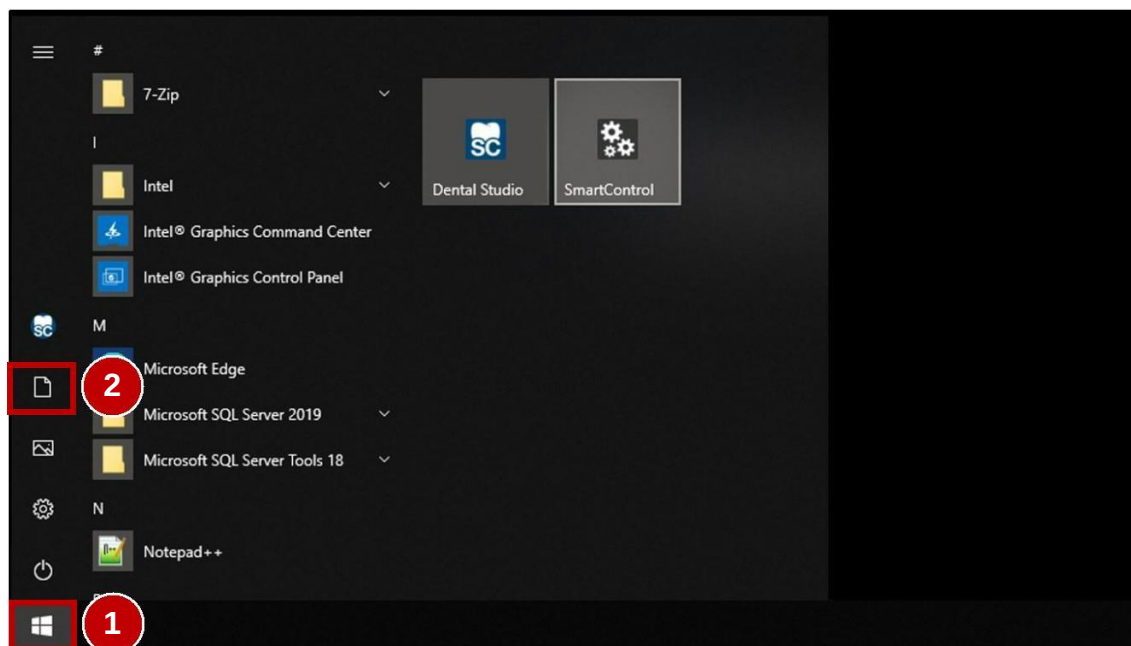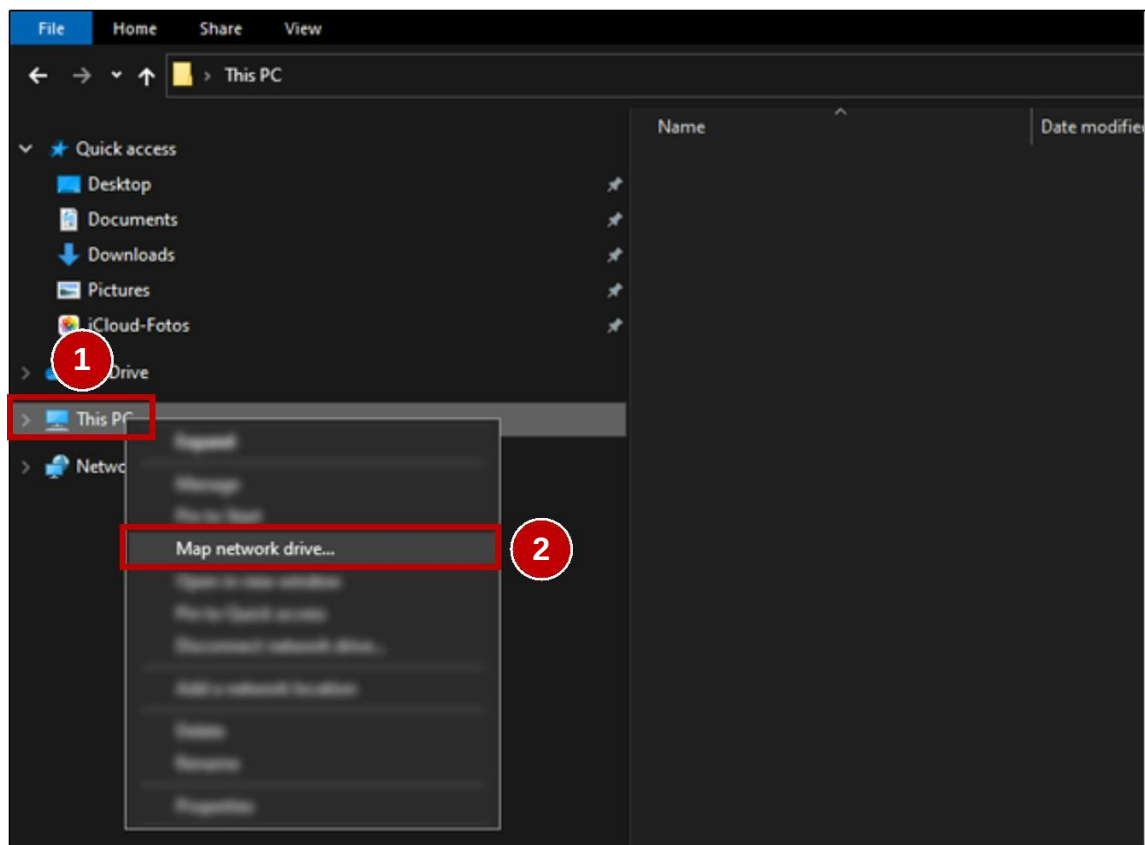
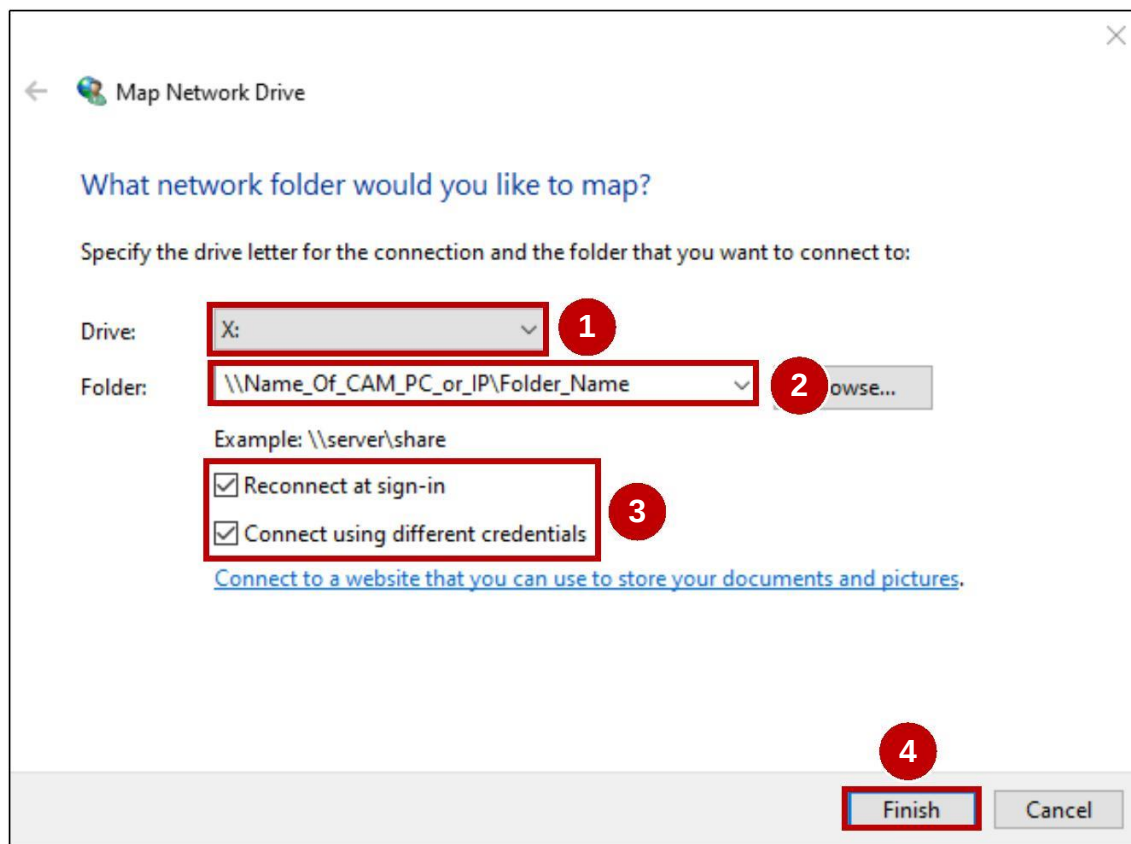
6.

1. Choose a Drive (1).
(For example X; Y; Z).
2. Type in the field Folder (2) the shared folder of the CAM PC.*(grey path of page 15)
3. Set both check marks (3).
4. Press Finish (4).

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Alternatively also the IP address possible (find out under Windows Icon--> cmd-->ipconfig write in black field and press Enter under IPv4 address : Enter "xxx.xxx.xxx.xx" as a replacement for Name_Of_CAM-OR_IP.

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Windows Security

Enter network credentials

Enter your credentials to connect to: Name_Of_CAM_PC_or_IP

Username 1

Password 2

Domain: [blurred]

☒ Remember my credentials 3

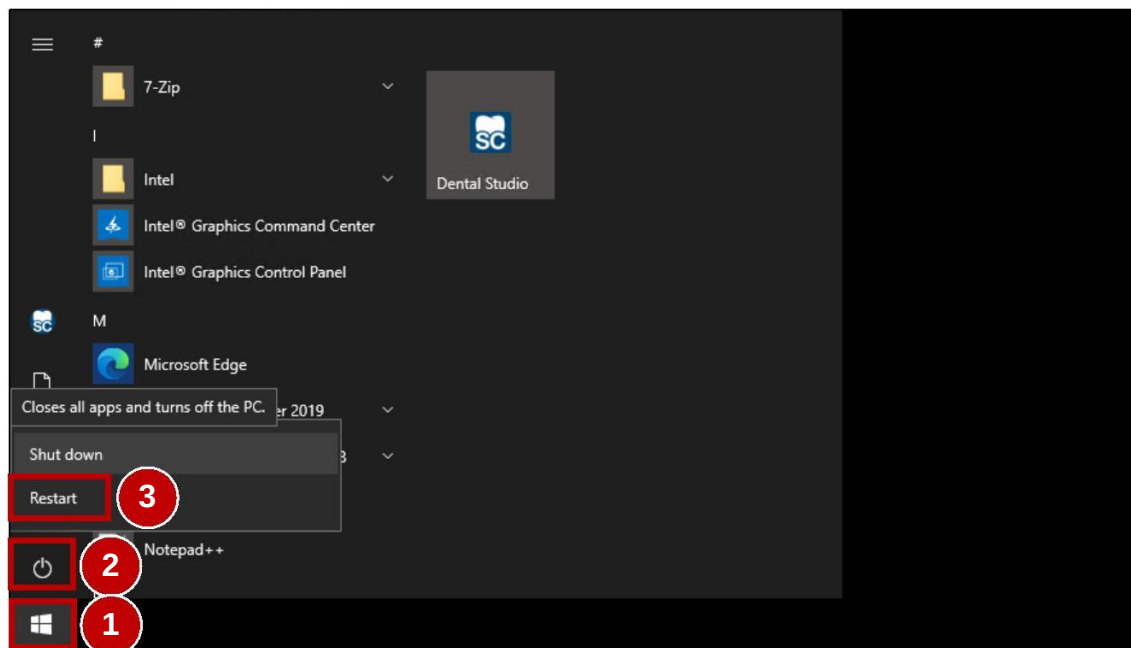
More choices 4

OK Cancel

1. Type in **icam** as Username (1).
2. Type in **imeS200-** as Password (2).
3. Set check mark (3).
4. Press OK (4).

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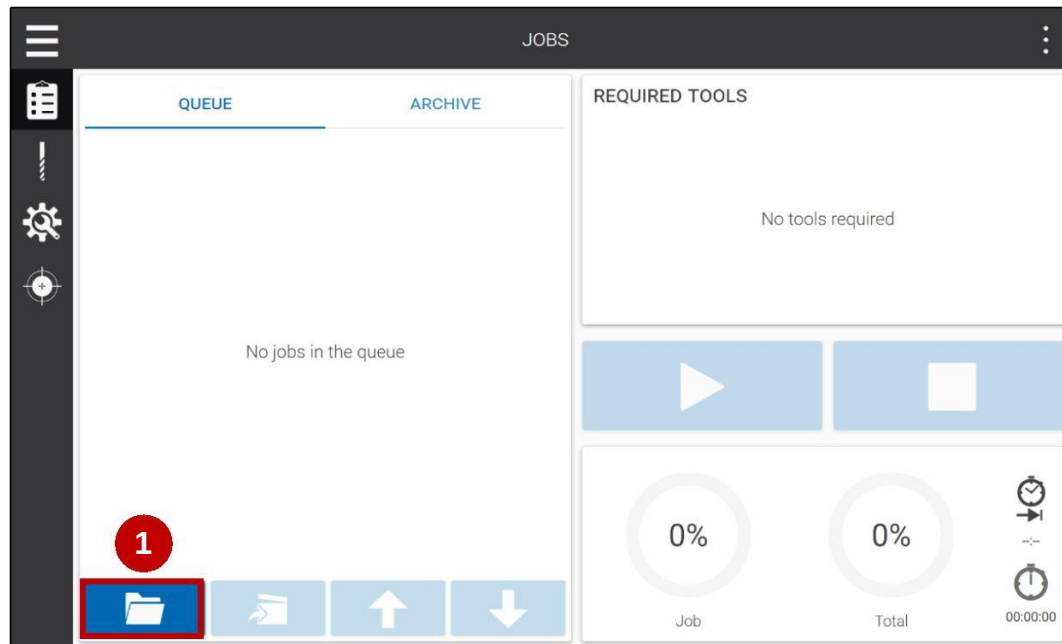
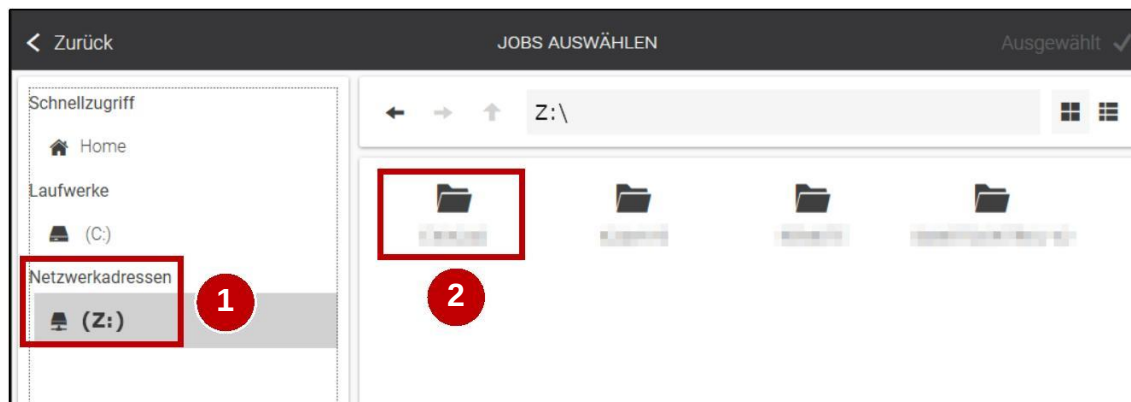
10.

1. Press the Network address tab (1).
2. Open the shared folder (2) from the CAM PC and select the corresponding milling file.
3. Network set-up accomplished!

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