E201003 37001 | E201003 37002

Brief instruction | Kurzanleitung

Connecting rechargeable power pack | Akkupack anschließen

CORiTEC 350i PRO

CORiTEC 350i Loader PRO

CORiTEC 350i PRO +

CORiTEC 350i Loader PRO +

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Information | Informationen

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The contents of this briefing introduction are intended exclusively for trading partners of imes-icore GmbH. A spreading of the guidance to third is forbidden!

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The following prerequisites must be ensured for the implementation of the content brief instruction:

- The acting person must be a qualified electrician!
 - The acting person must have received a dealer training by imes-icore GmbH!
 - Use suitable personal protective equipment!
 - All contents of the included documents (instructions for use, accessories, etc.) must be read and understood!
-

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Copyright | Urheberschutz

The contents of this brief instruction are copyright protected and are the intellectual property of imes-icore GmbH. Usage of this content is only permissible for the operation of the machine. Any use outside of this is prohibited without the express written permission of the manufacturer. All rights, also those of translation, are reserved.

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Trademark protection | Markenschutz

All rights to product, company and brand names or third-party rights in this brief instruction belong to the respective company or holder, regardless of their form, and are subject to international copyright and trademark laws. Individual labelling has been omitted from this brief instruction. imes-icore® is a legally protected trademark according to section 4 no.1 MarkenG (trademark law).

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Protective work clothing, protective gloves, safety footwear, respiratory protection, safety goggles

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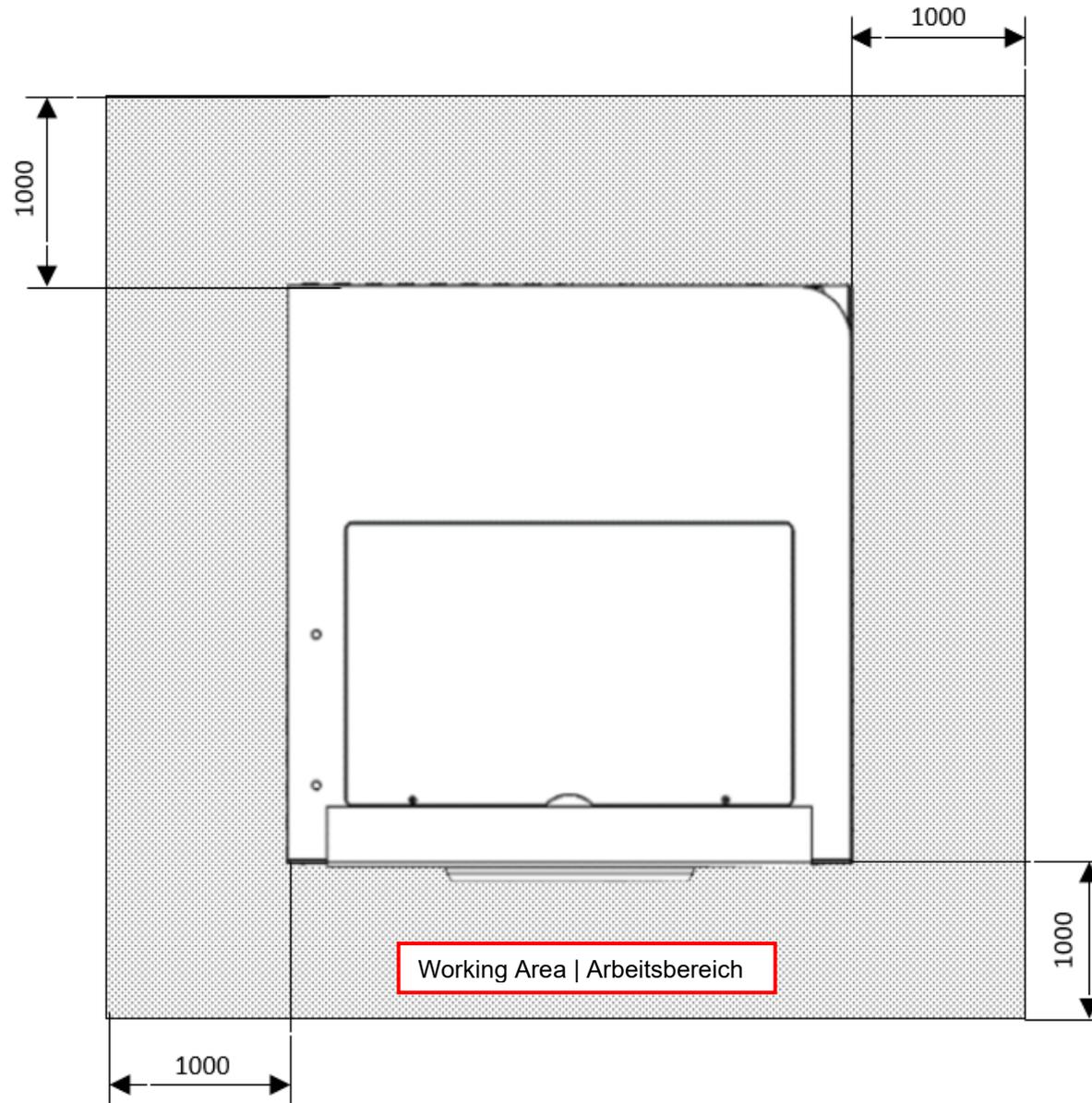
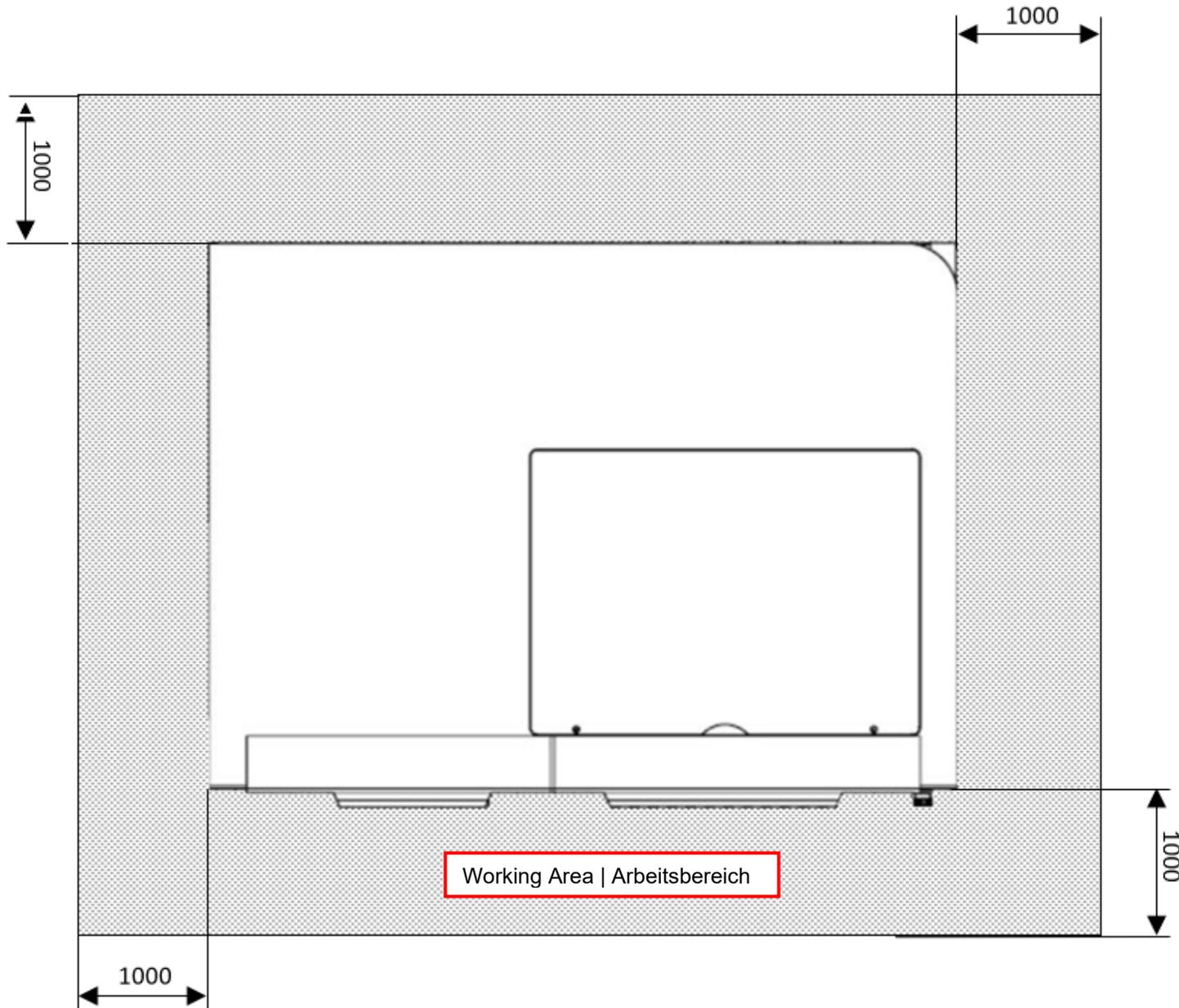
Safety instructions | Sicherheitshinweise

⚠ DANGER

Danger to life due to improper handling, repair or maintenance!

Improper handling, repair or maintenance will result in death, life-threatening injuries or serious damage to the machine!

- Work on or with the machine must be carried out by qualified and trained personnel!
- Work on or with the machine must always be carried out strictly in accordance with the manufacturer's documentation!
- Only use original spare parts from the machine manufacturer
- Always wear suitable personal protective equipment according to the work to be carried out!

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⚠ DANGER

Danger to life due to electric shock!

The machine has mains voltage! Touching live parts will result in death or serious injury due to electric shock!

- All maintenance and repair work may only be carried out by qualified electricians!
- All maintenance and repair work may only be carried out in currentless state of the machine!
- Remove the power cord and wait a few minutes before you start work!

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⚠ WARNING

Risk of injury from rotating tools and moving axles!

Contact with moving parts of the machine can cause serious injury!

- Always be careful and follow the general safety rules!
- Never reach into working machinery!
- Wear protection gloves!

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⚠ CAUTION

Risk of injury by swarf!

Swarf in the machine may have sharp edges and cause deep lacerations!

- Clean the machine thoroughly, always before starting any work!
- Wear protection gloves and safety goggles!

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
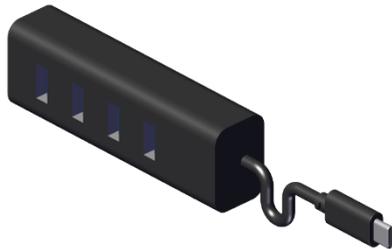
Before starting repair- and maintenance work, the current condition of the machine must be checked and compared with its delivery condition. In particular, the operability of safety components must be checked here. Existing defects and manipulated components must be repaired as quickly as possible. In addition, existing defects must be documented and photographed.

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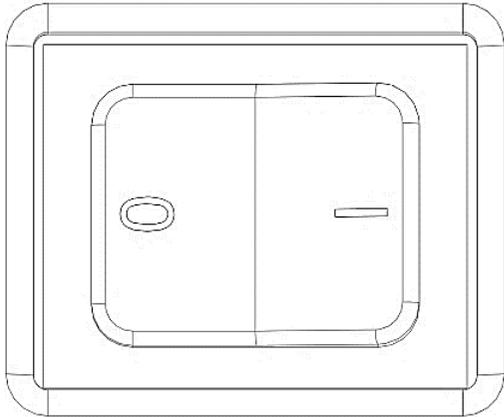1.


Switch on the machine at the main switch.

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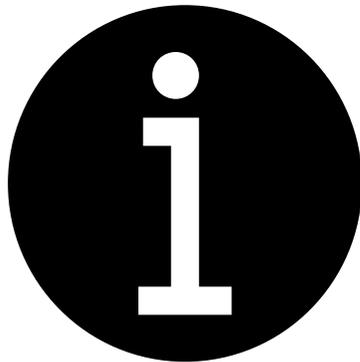2.


Save the folder "C:\CNC Workbench" on an USB stick.

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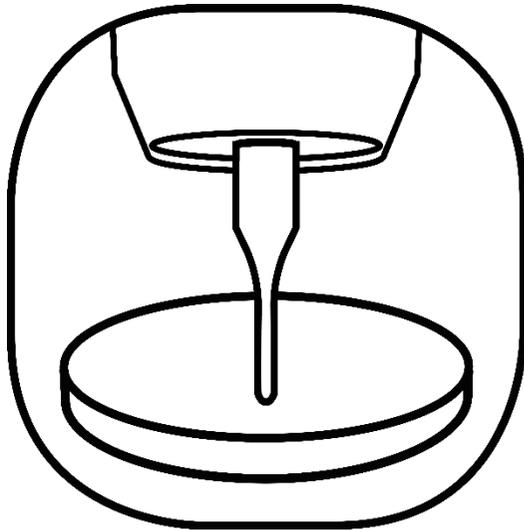3.


Start "Remote Dental" and perform a reference run.

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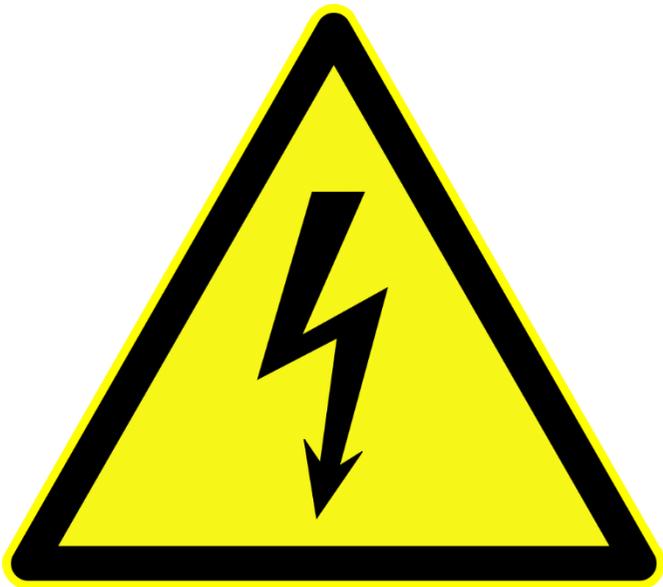4.



The following work is carried out with the machine switched on in the vicinity of live components. The regulations of the respective country where the machine is located must be observed! The work must be carried out by a qualified electrician!

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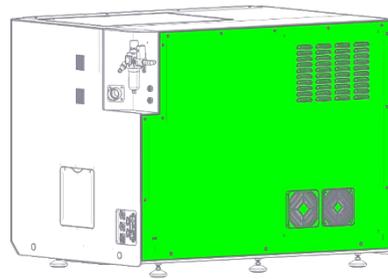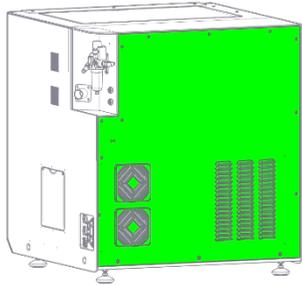5.

Remove the rear panel to gain access to the control cabinet (see service manual).

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
Connecting rechargeable power pack | Akkupack anschließen

1.

The following work must be carried out using ESD equipment.

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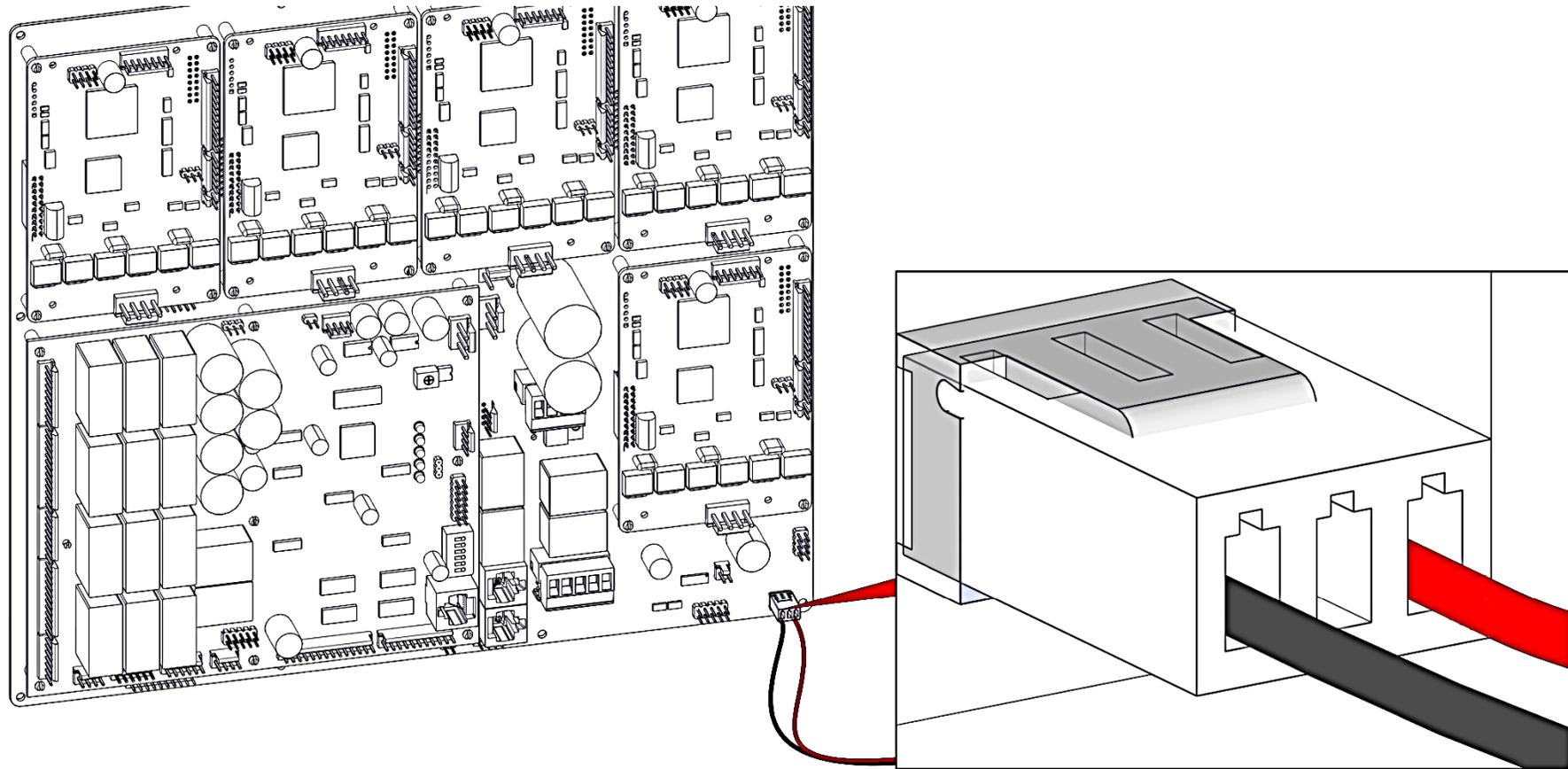2.

Disconnect the plug of the old battery pack from the main board of the control unit.

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3.

Remove the old battery pack from the machine.

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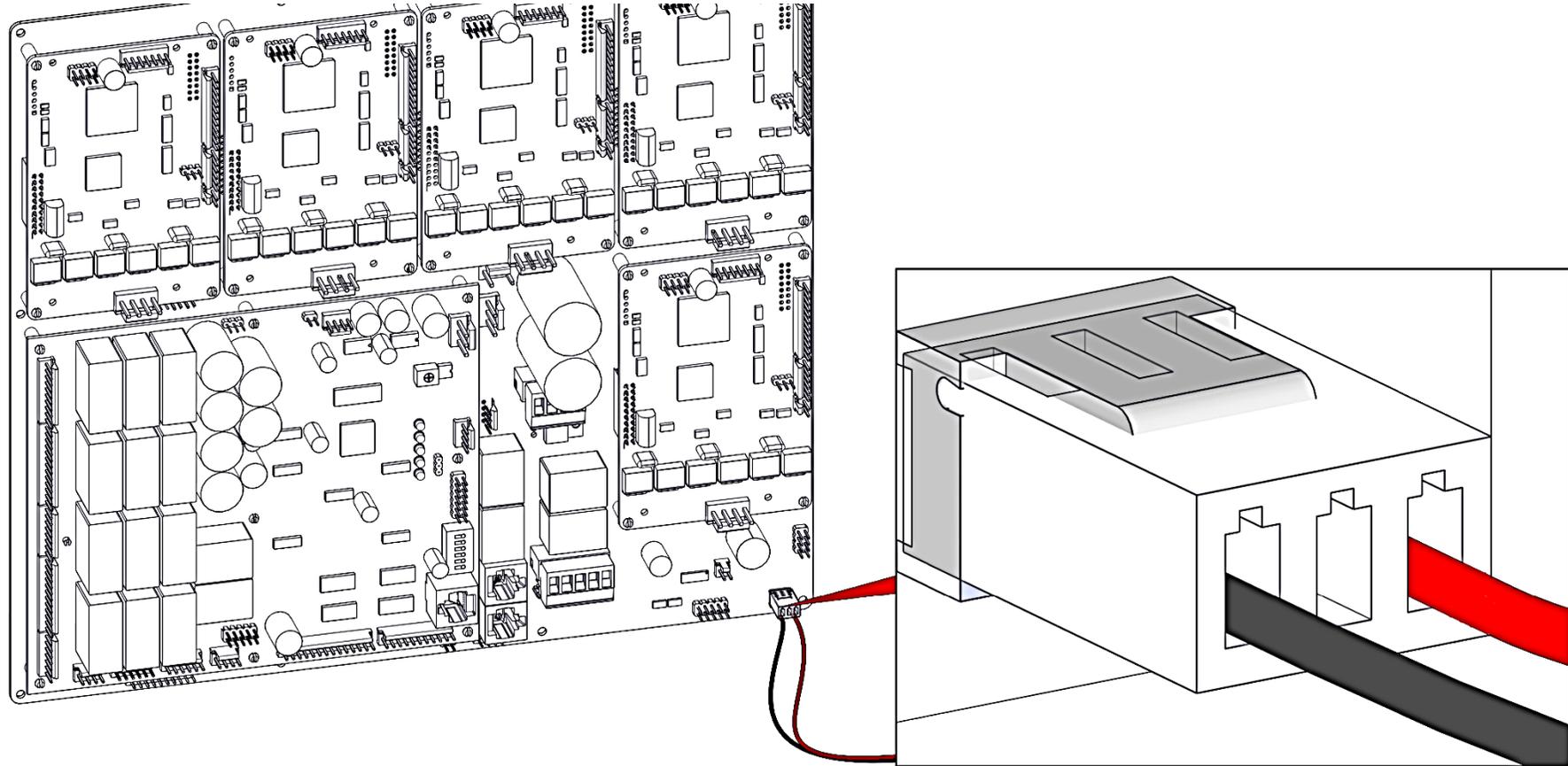
4.

Stick the fleece tape onto the rechargeable power pack. Attach the new rechargeable power pack to the Velcro strap of the old battery pack in the machine.

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
5.

Connect the plug of the rechargeable power pack to the main board of the control unit. The three-pole contact of the old battery pack is used for this purpose.

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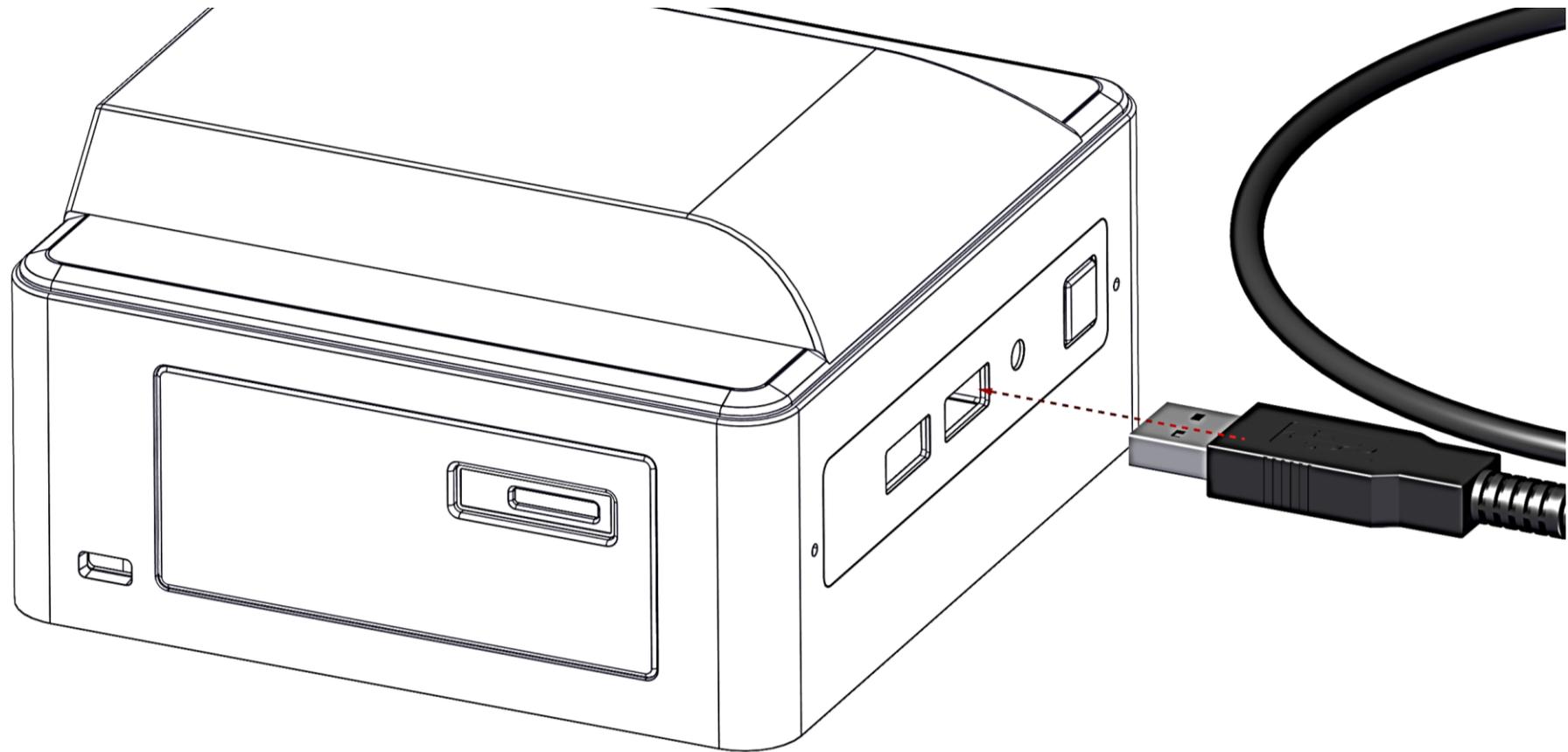
6.

Connect the USB plug of the rechargeable power pack to the computer. With the 350i PRO + and 350i Loader PRO +, a USB hub must also be installed so that all USB ports on the computer are occupied.

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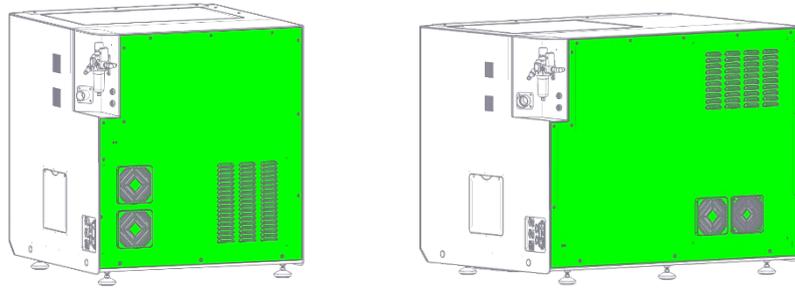2.

Measure insulation resistance, protective resistance and the equivalent current! The measurement results must then be recorded. If necessary, existing defects must be eliminated.

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imes-icore® GmbH