Brief Instruction | Kurzanleitung

CORiTEC Systems

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Information | Informationen

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The contents of this Brief instruction are intended exclusively for trading partners of imes-icore GmbH. A spreading of the guidance to third is forbidden!

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Copyright | Urheberschutz

The contents of this Brief instruction are copyright protected and are the intellectual property of imes-icore GmbH. Usage of this content is only permissible of using the machine. Any use outside of this is prohibited without the express written permission of the manufacturer. All rights, also those of translation, are reserved.

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Trademark protection | Markenschutz

All rights to product, company and brand names or third-party rights in this operating manual belong to the respective company or holder, regardless of their form, and are subject to international copyright and trademark laws. Individual labelling has been omitted from this Brief instruction. imes-icore® is a legally protected trademark according to section 4 no.1 MarkenG (trademark law).

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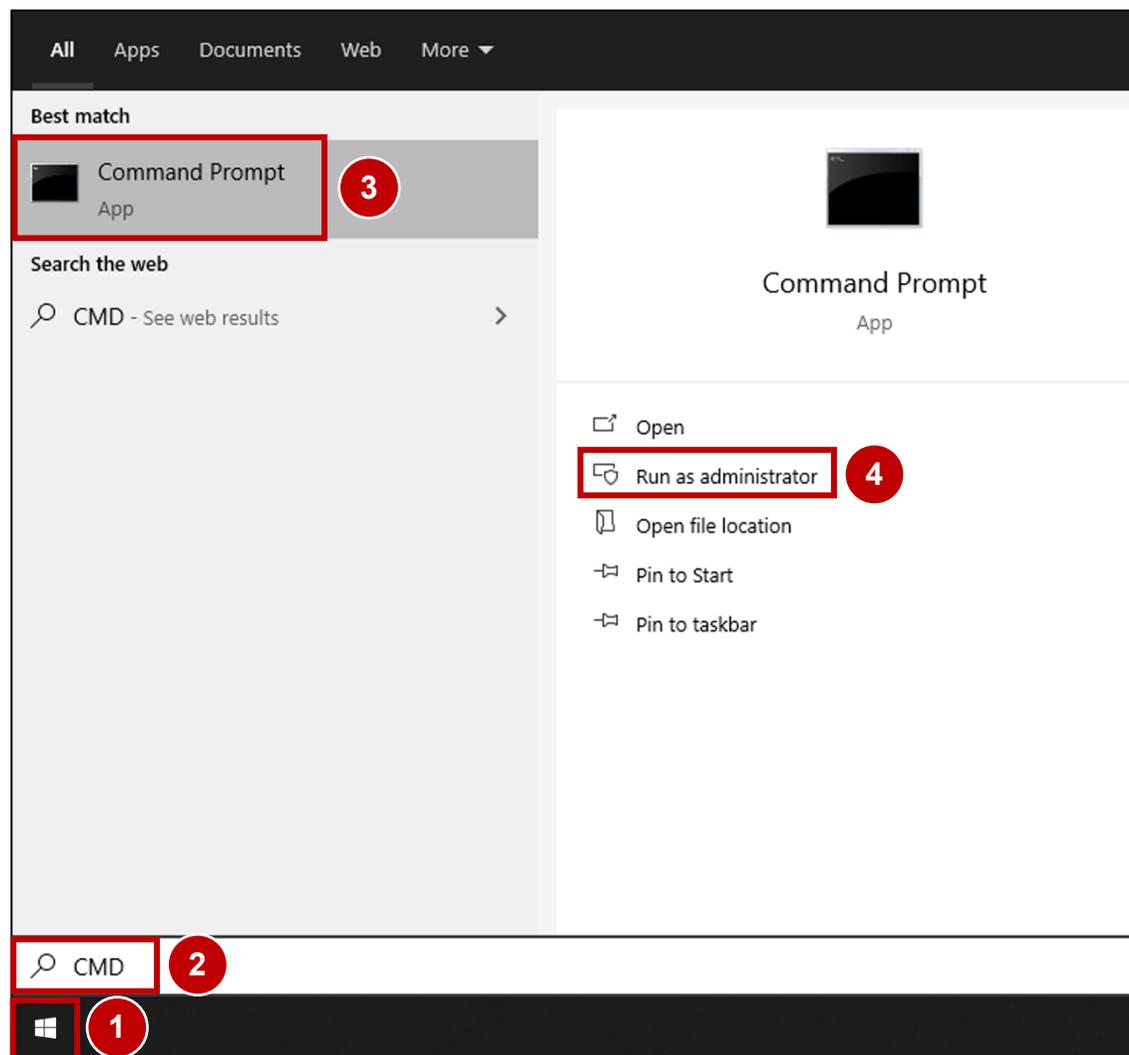
Transport, installation and commissioning of the machine according to the operating manual! Observe the warning and safety instructions in the operating manual!

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
1.

1. Switch on the CAM PC at the main switch.
2. Wait until the CAM PC has completely booted.
3. Press the Windows® Start button (1).
4. Type **cmd** (2) to search the Command Prompt App (3).
5. Press Run as administrator (4).

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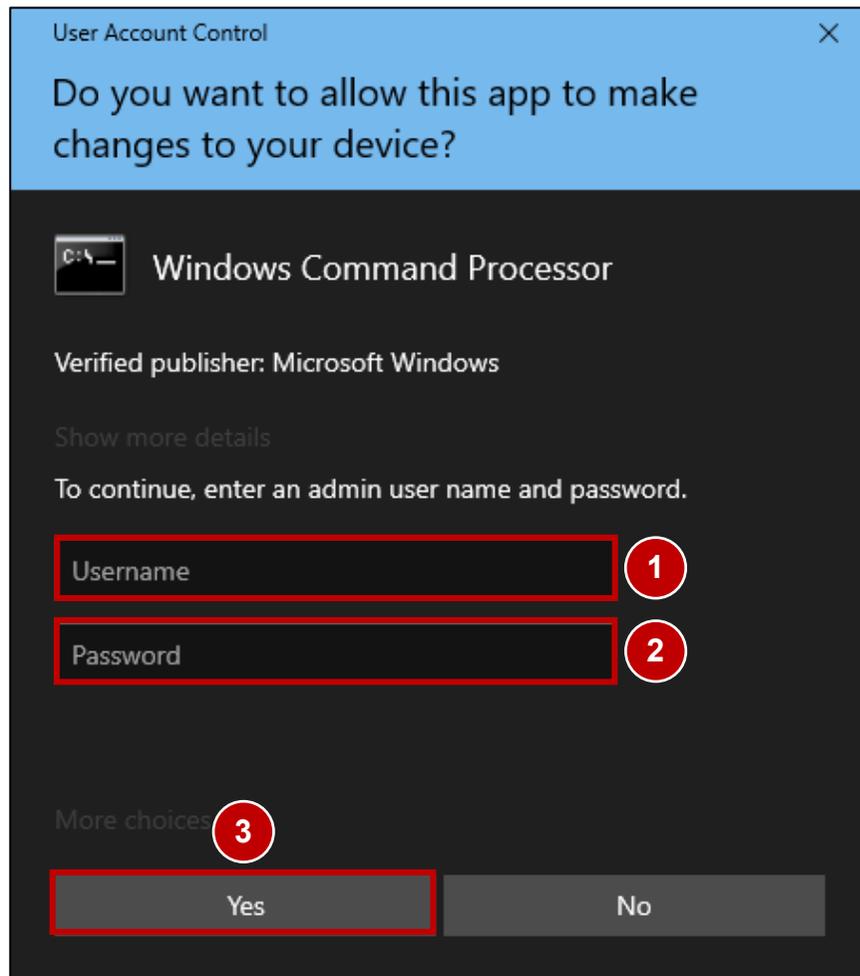2.

1. Type in the user name with admin rights (1).
2. Type in the Password (2) for the user.
3. Press Yes (3).

Note: For customised CAM-PCs which are not delivered by imes-icore, the corresponding user name and password must be requested from the customer!

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3.

```
Administrator: Command Prompt
Microsoft Windows [Version 10.0.19042.685]
(c) 2020 Microsoft Corporation. All rights reserved.
C:\WINDOWS\system32> net user "icam" "imeS200-" /add
```

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1. Type in **net user "icam" "imeS200-" /add** (1) and press the Enter key.

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4.

```

Administrator: Command Prompt
Microsoft Windows [Version 10.0.19042.685]
(c) 2020 Microsoft Corporation. All rights reserved.

C:\WINDOWS\system32>net user "icam" "imeS200-" /add
The command completed successfully. 1

C:\WINDOWS\system32>exit 2

```

1. If the Text (1) – The command completed successfully – appears the network user was created successfully!
Note: If the Text – The user already exists – appears skip to next Page!
2. Type in **exit** (2) and press the Enter key.

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5.


```

Administrator: Command Prompt
Microsoft Windows [Version 10.0.19042.685]
(c) 2020 Microsoft Corporation. All rights reserved.

C:\WINDOWS\system32>net user "icam" "imesS200-" /add
The account already exists.

More help is available by typing NET HELPMSG 2224.

C:\WINDOWS\system32>exit

```

1. If the Text (1) – The user already exists – appears the network user already was created once before!
2. Type in **exit** (2) and press the Enter key.
3. Check if the User name (icam) and password (imesS200-) are working otherwise you have to remove the User icam manually!
(Contact your Network Administrator or the imes-icore customer service!)
4. After removing the User icam repeat the process *Create network user | Netzwerkbenutzer anlegen!*

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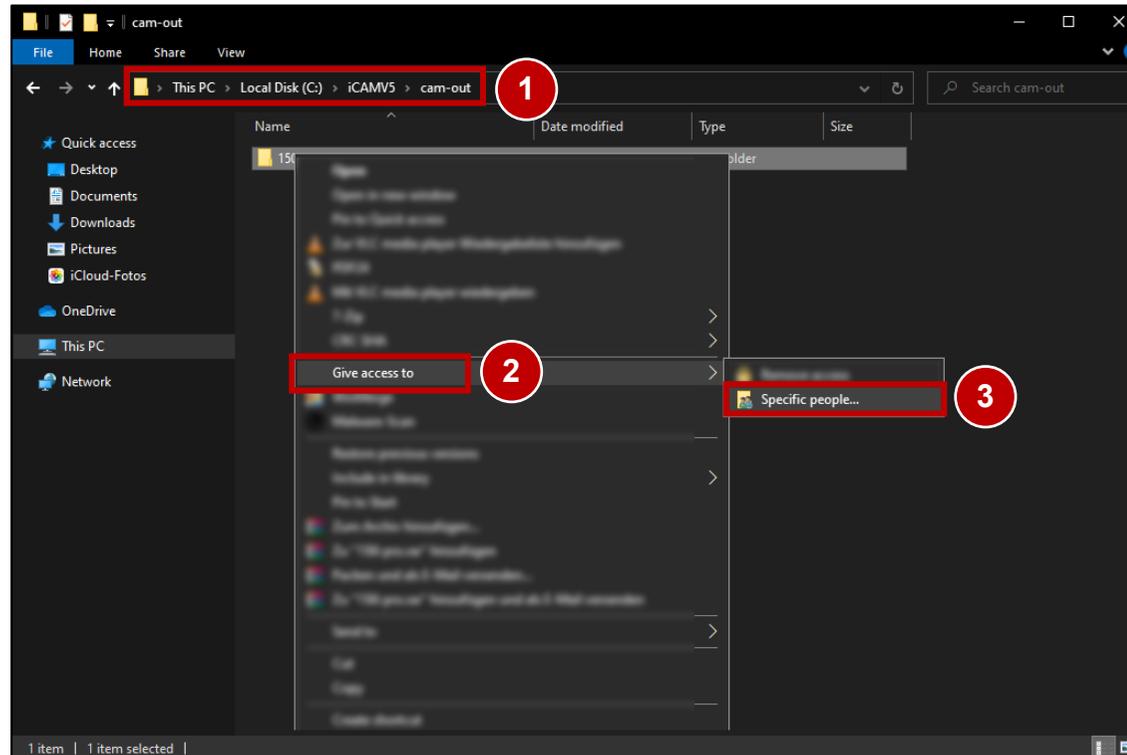
1.

1. Open a new Windows® Explorer window. (Windows® button + E)
2. Navigate (2) to the CAM output folder.
3. Right-click on the folder to be shared in the network.
4. Open Give access to (2).
5. Press Specific people (3).

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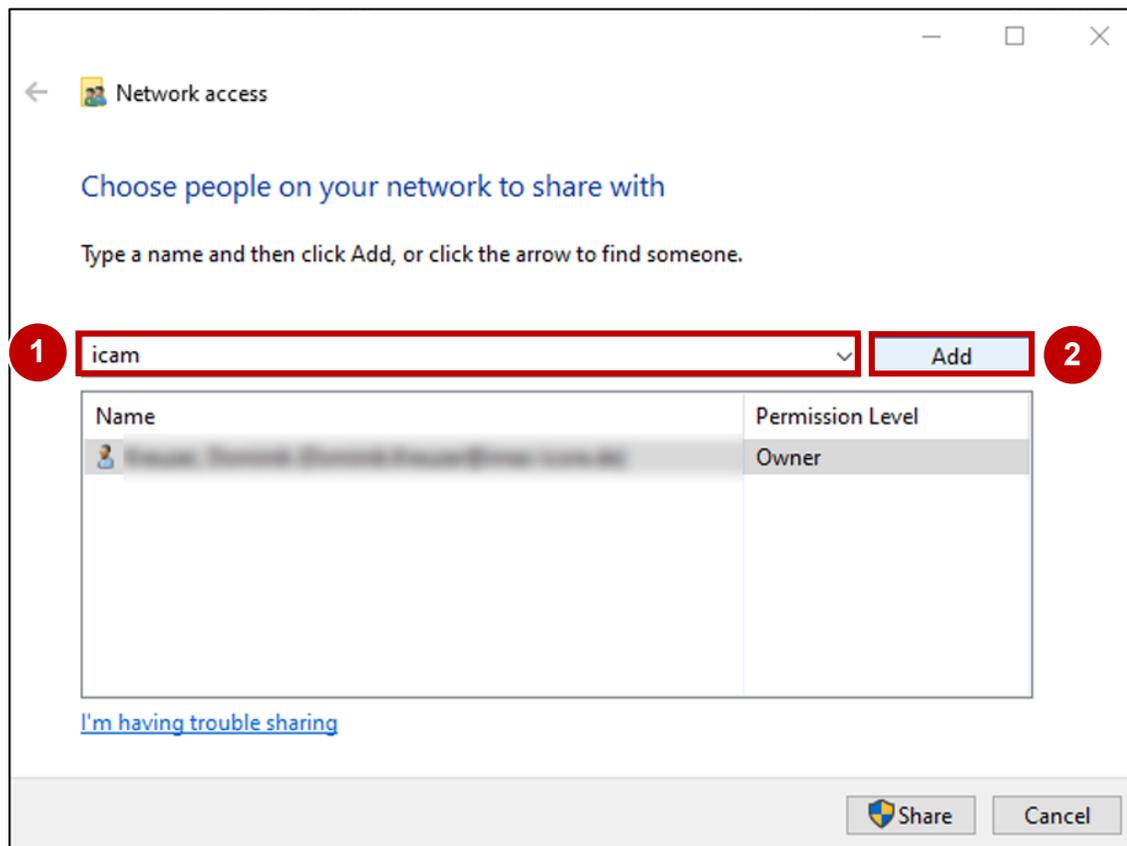
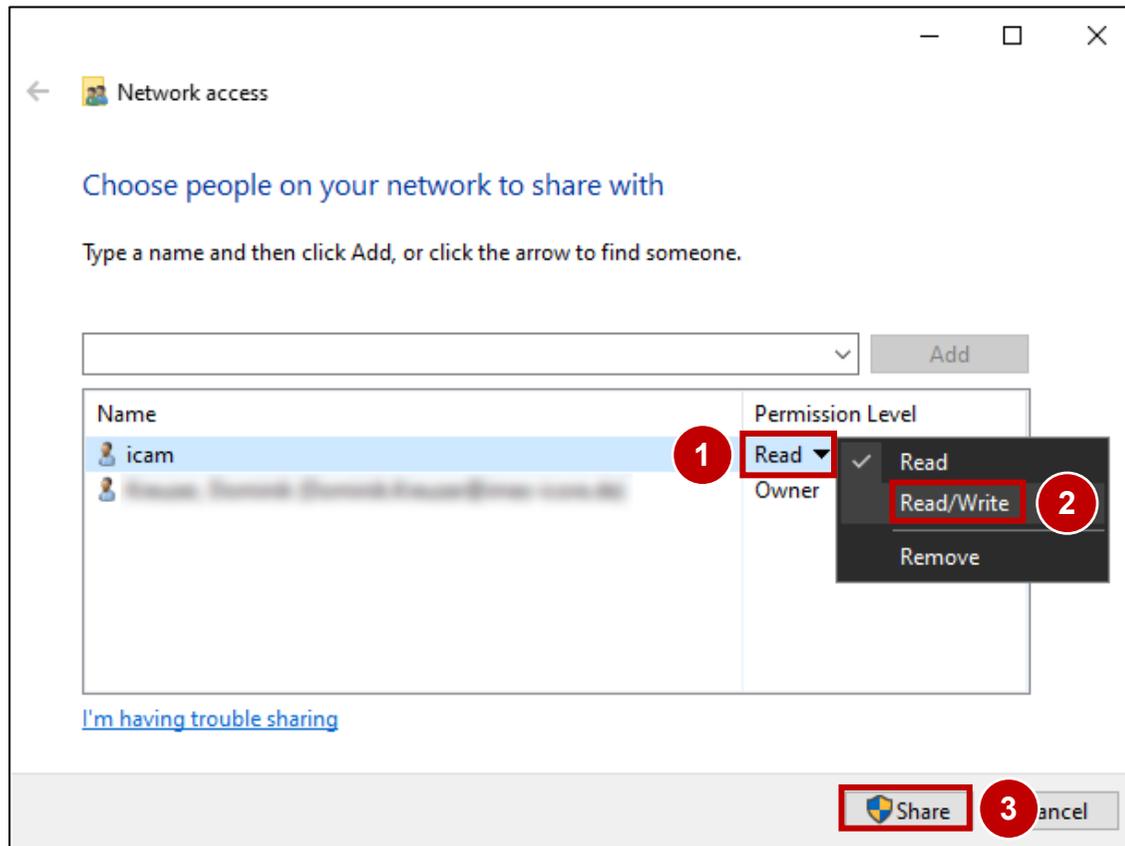
3.

1. Press onto the Permission Level (1) of the User icam.
2. Choose Read/Write (2).
3. Press Share (3).

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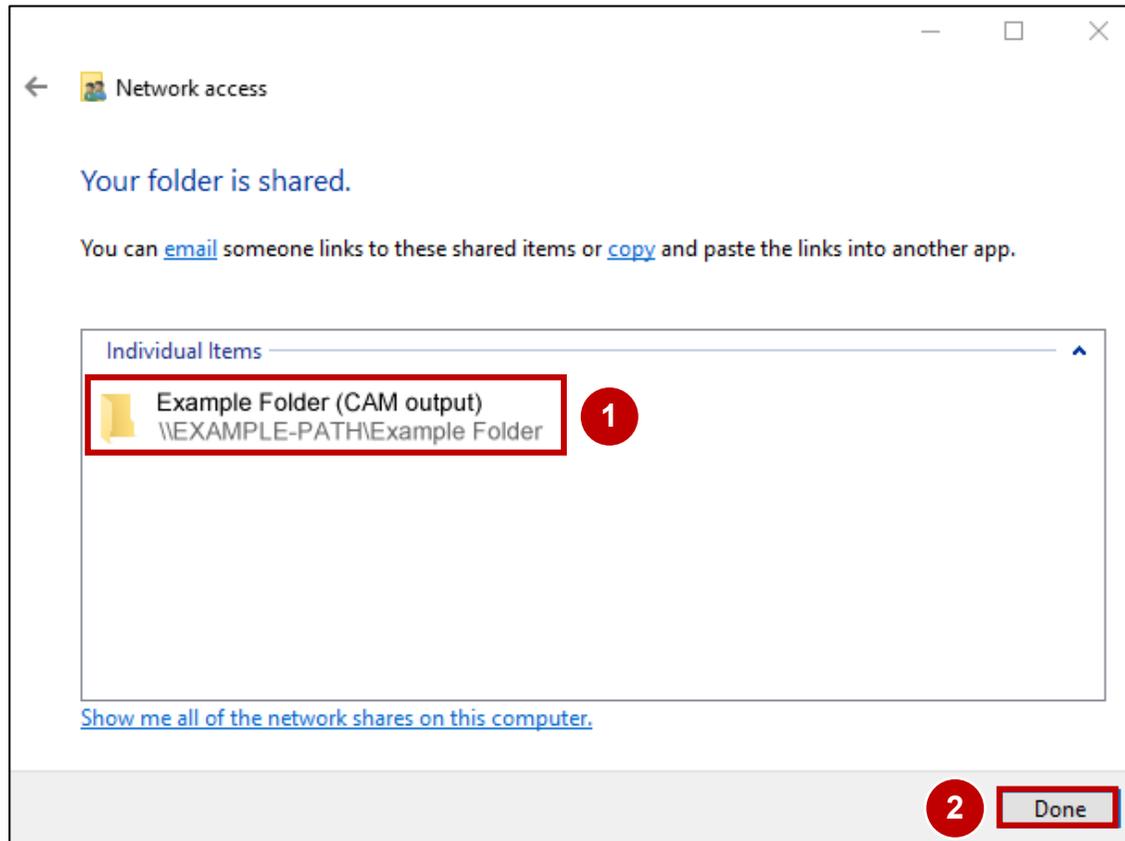4.


1. Check the correctness of the shared folder (1).
2. Press Done (2).
3. The network set-up of the CAM PC is accomplished!

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CORiTEC system with SmartControl V2.0 | CORiTEC System mit SmartControl V2.0

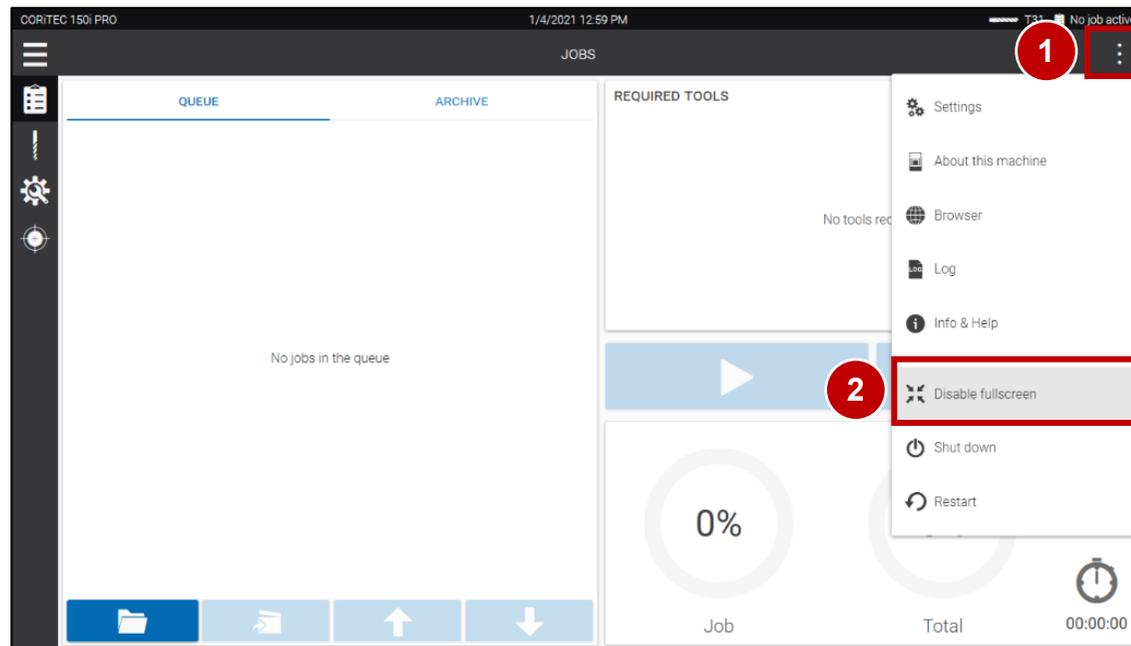
1.

1. Switch on the machine at the main switch.
2. Wait until the Customer UI has completely booted.
3. Press the menu button (1).
4. Press Disable fullscreen (2).

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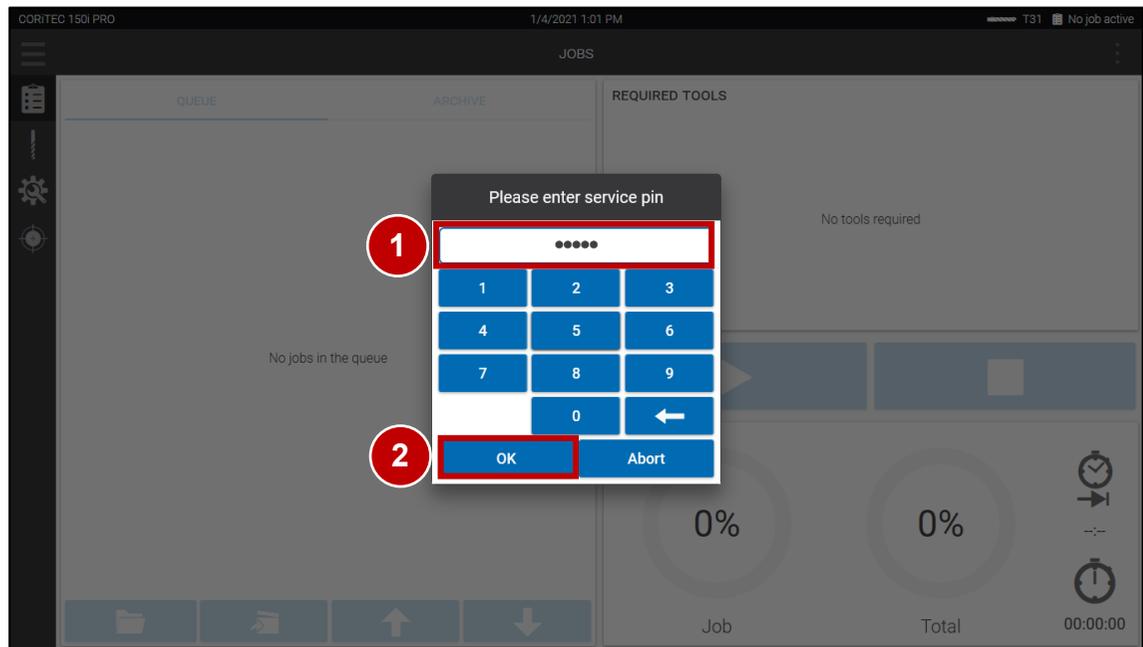
2.

1. Enter Password (1).
(Note: To receive Password contact imes-icore customer service!)
2. Press OK (2).

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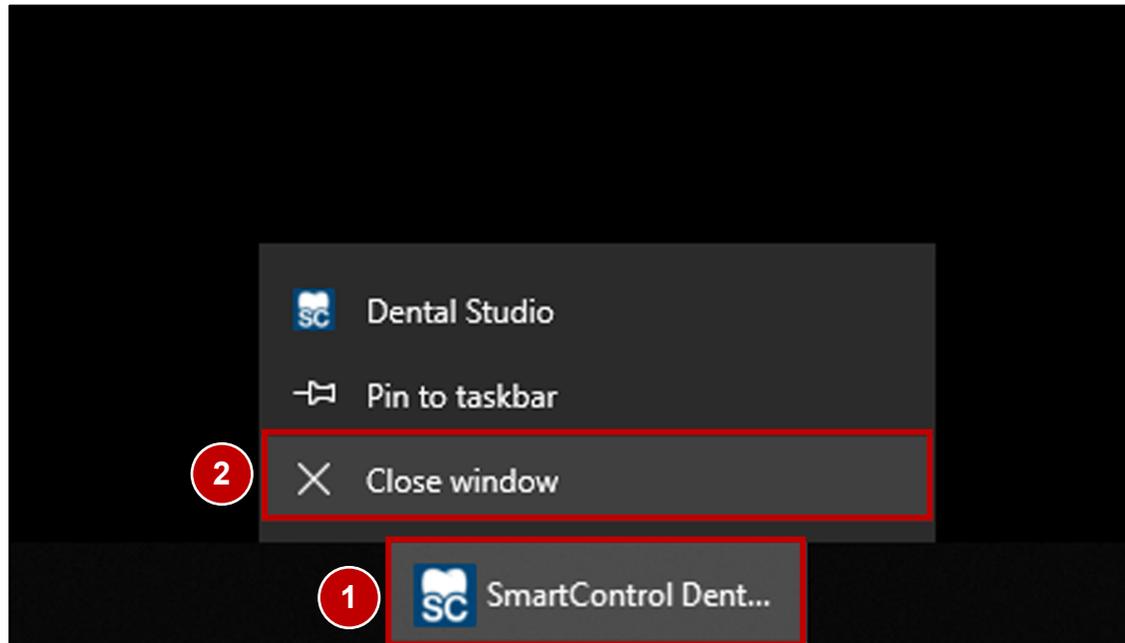
3.

1. Press on and hold on the minimized SmartControl task (1) in the Taskbar to open the context menu.
2. Press Close window (2) to quit the Customer UI.

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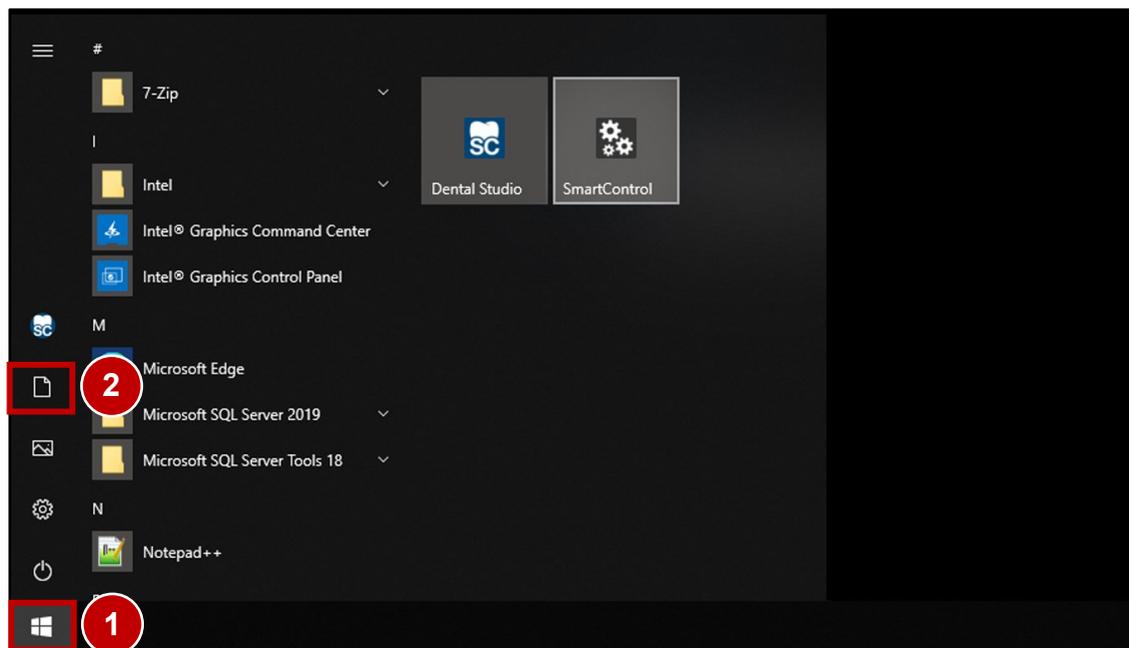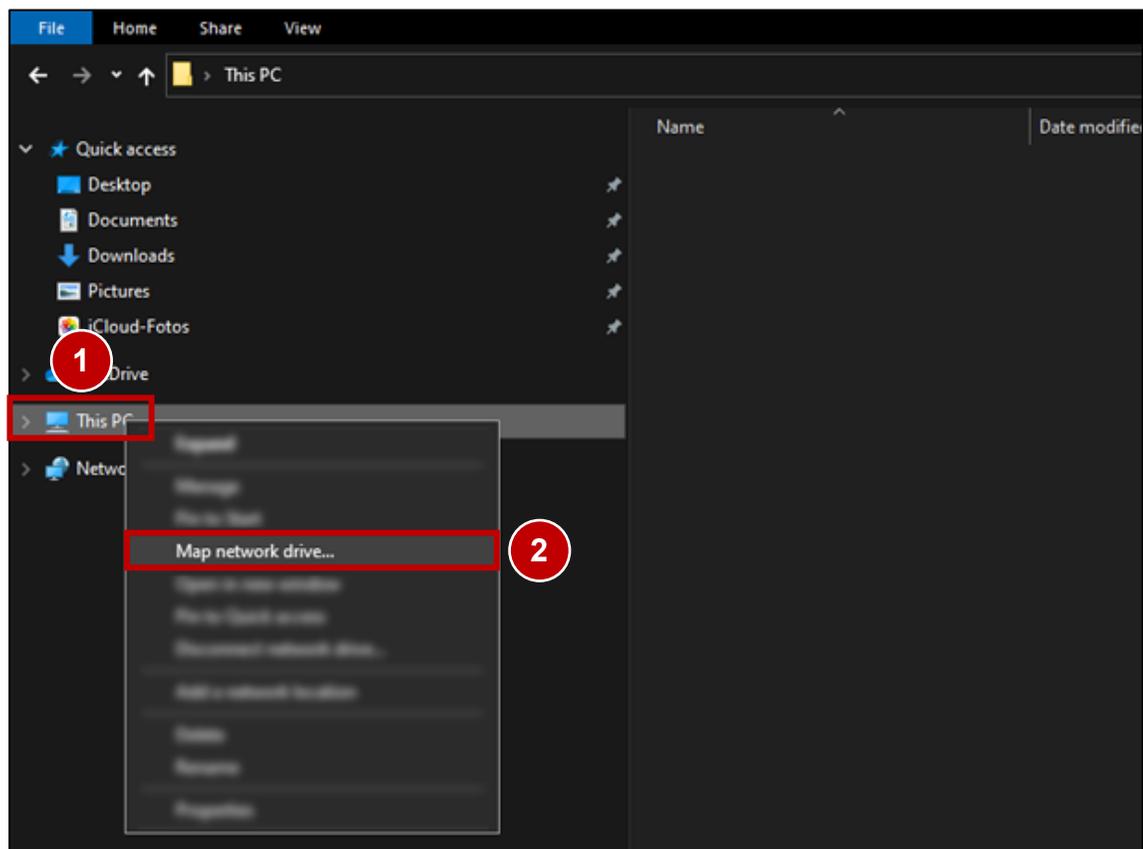
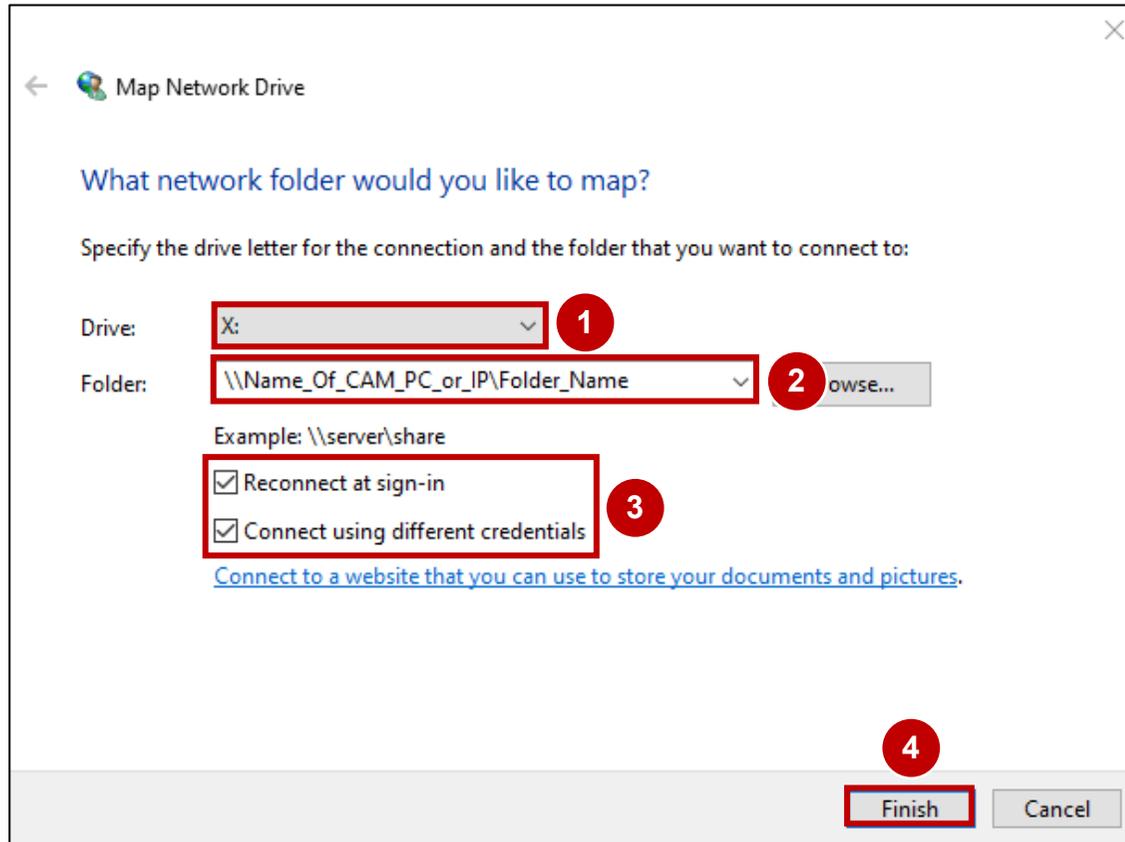
6.

1. Choose a Drive (1).
(For example X; Y; Z).
2. Type in the field Folder (2) the shared folder of the CAM PC.
3. Set both check marks (3).
4. Press Finish (4).

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
7.

The image shows a Windows Security dialog box titled "Enter network credentials". The dialog prompts the user to enter credentials to connect to a device named "Name_Of_CAM_PC_or_IP". It features four input fields, each highlighted with a red box and a red circle containing a number: 1. Username, 2. Password, 3. "Remember my credentials" checkbox, and 4. "More choices" link. At the bottom, there are "OK" and "Cancel" buttons, with the "OK" button also highlighted with a red box.

1. Type in **icam** as Username (1).
2. Type in **imeS200-** as Password (2).
3. Set check mark (3).
4. Press OK (4).

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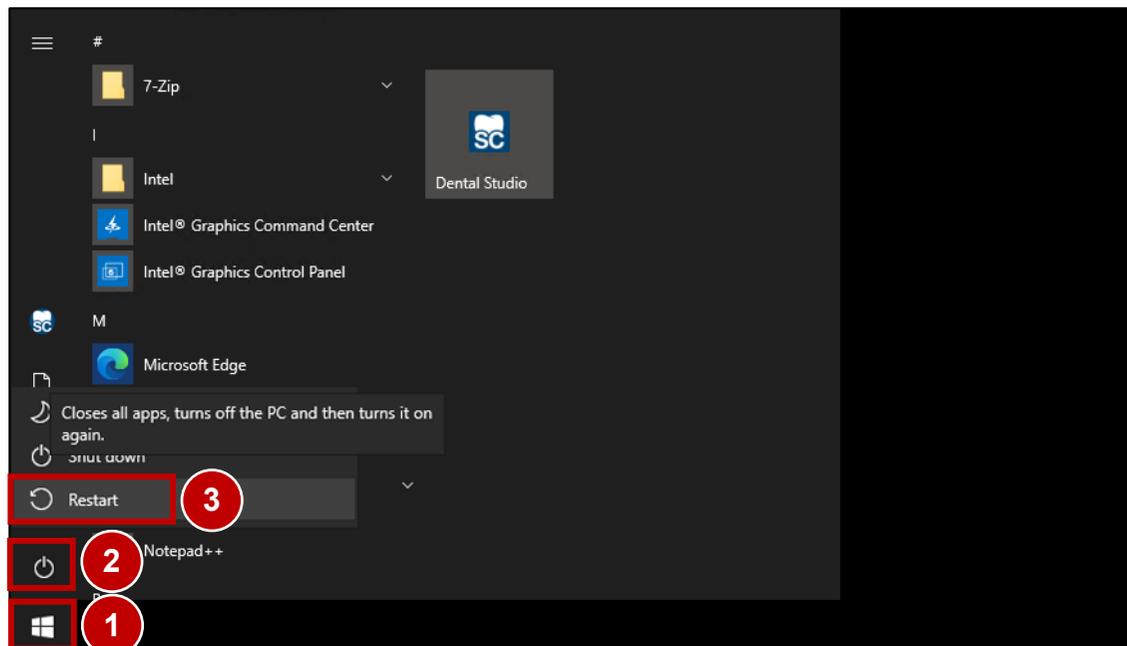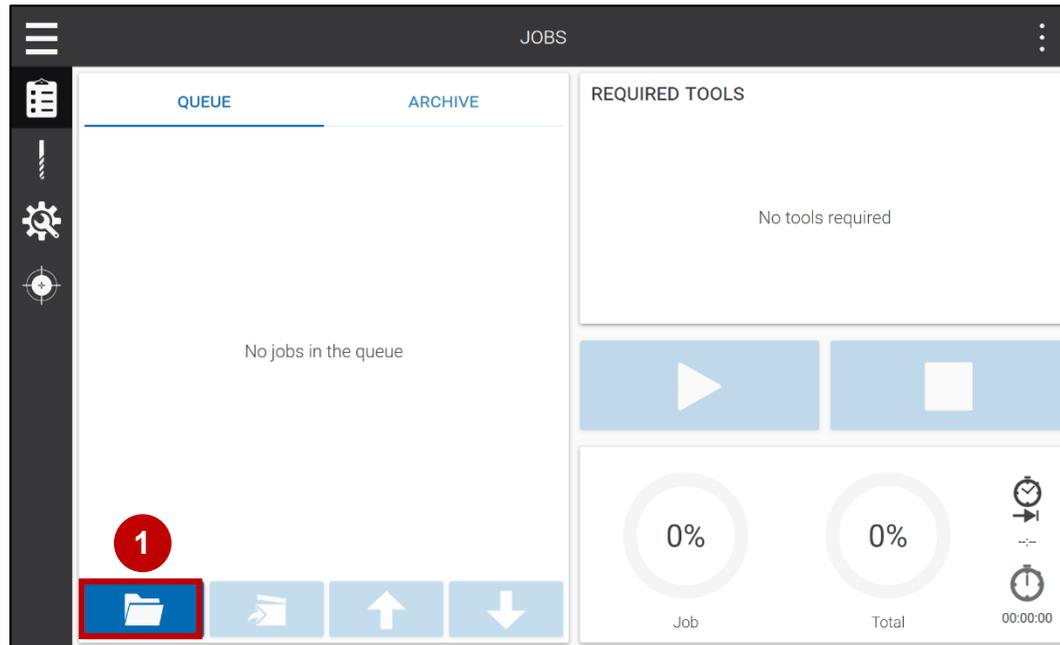
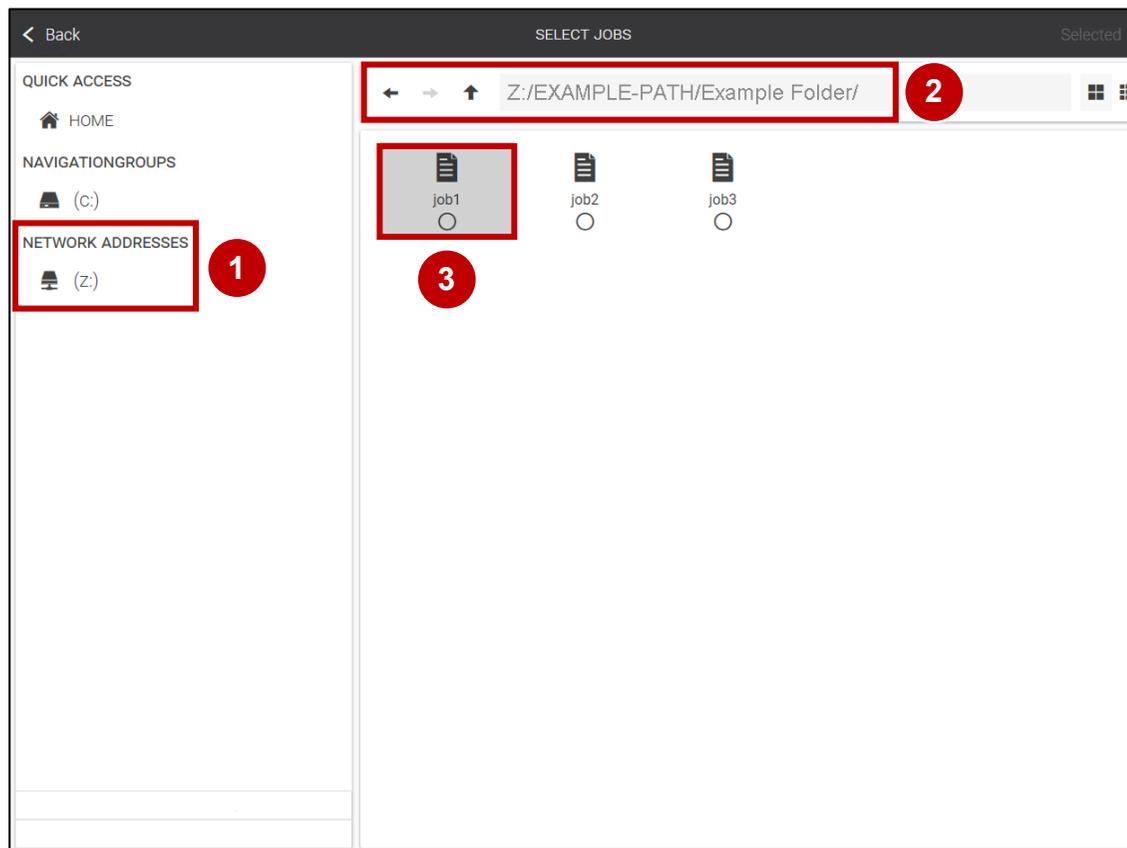
10.


1. Press the Network addresses tab (1).
2. Open the shared folder (2) of the CAM PC.
3. Select the corresponding milling file (3).
4. Network set-up accomplished!

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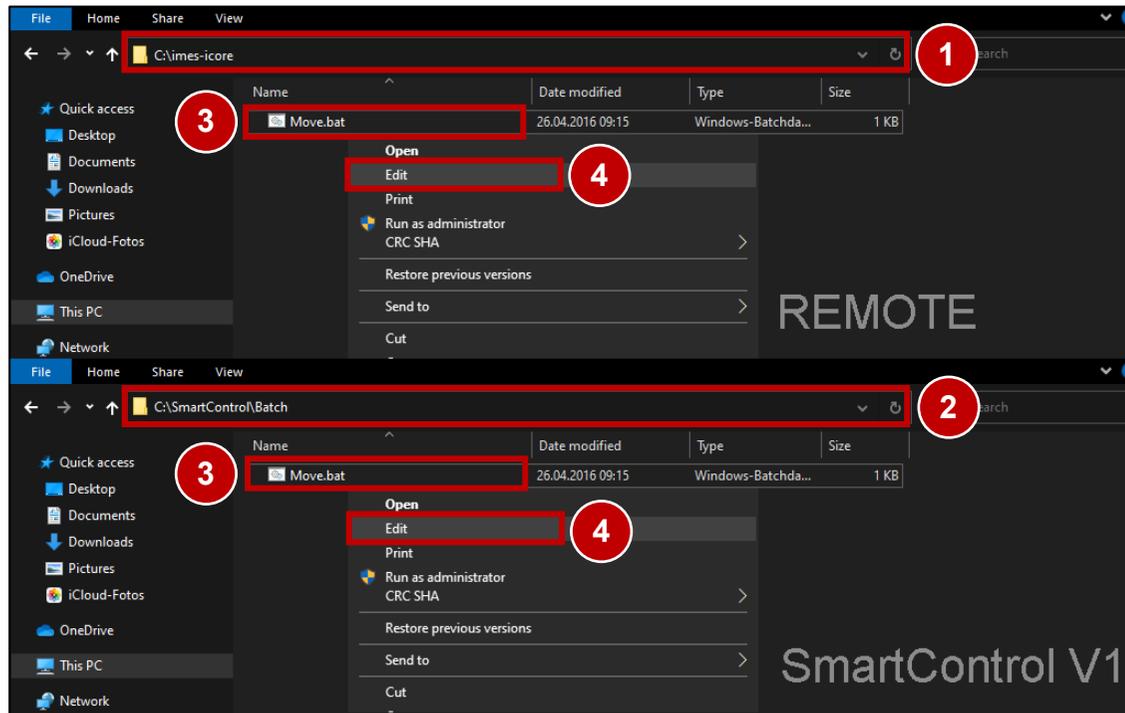1.

1. Switch on the Machine at the main switch.
2. Wait until the Machine has completely booted.
3. Do not start REMOTE or SmartControl V1 otherwise close the Software!
4. Open Windows® Explorer.
5. Navigate to Folder:
(1) For Machine with REMOTE
C:\imes-icore\
(2) For Machine with SmartControl V1
C:\SmartControl\Batch\
6. Right-click the file Move.bat (3) to open the context menu.
7. Press Edit (4).

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1. Copy and paste the text from the picture into the Move.bat or insert it manually.
2. Replace the red marked text **EXAMPLE-PATH\\Example Folder** with the correct shared CAM PC output path!
3. Replace the red marked text **NC_Daten\\XXXi** with the correct Machine path!
Note: Observe correct writing!
4. Press File.
5. Press Save.
6. Close the Editor.
7. Network set-up accomplished!

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