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<div><div></div>Instructions for use</div>	<div><div></div>May 1, 2023 DMS-IFU-LD (Rev.7)</div>
EN English	
<div><div></div>Indication</div> <div>Coloring liquid is a water-based solution for coloring zirconia dental restorations close to natural teeth color. Apply before sintering using brush or dipping method.</div>	
<div><div></div>Intended Use</div> <div>A color solution used to color ceramic dental prostheses, such as crowns and bridges, to match the color of the individual patients.</div>	
<div><div></div>Intended User</div> <div>Dentist, Dental technician</div>	
<div><div></div>Target Patient Group</div> <div>Patients in need of prosthetic treatment due to tooth loss and caries</div>	
<div><div></div>Storage Condition</div> <div>The remaining solution after use should be closed and stored in a dry place at 2~ 28°C and avoid direct sunlight.</div>	

<div><div></div>Warnings</div> <div>1) This product is for professionals such as dentist or dental technicians; the use of material other than intended users is strictly prohibited. <p>2) Do not use other than the intended use</p> <p>3) Do not use in case of contaminants, such as impurities are present.</p> <p>4) Familiarize yourself with how to use the product before use.</p></div>	
<div><div></div>General Precautions</div> <div>1) Do not eat or smoke when applying coloring. <p>2) Wear protective gloves, glasses and gowns when using the product.</p> <p>3) In case of contact with skin, wash with water thoroughly and if in eyes, rinse thoroughly with water and consult a physician.</p></div>	

<div><div></div>Procedural Precautions</div> <div>1) Do not leave the cover open for a long time to prevent contamination. <p>2) Do not use brushes and trays made of metal as they may cause contamination of the solution.</p> <p>3) Used on zirconia restorations and needs sufficient drying after coloring.</p> <p>4) Be sure to wear gloves and wash quickly when in contact with skin.</p> <p>5) Do not use when there are contaminants such as impurities.</p> <p>6) Only specialists who have received regular training on prosthetic procedures should perform this procedure.</p> <p>7) Be careful not to get the coloring into eyes and mouth when working.</p></div>	
---	--

<div><div></div>Preparation before use</div> <div>1) Read and follow the manufacturer's instructions. <p>2) Do not use expired products.</p> <p>3) Do not use if packaging damage or product abnormality is found.</p> <p>4) Check the model name of the product before use.</p></div>	
--	--

<div><div></div>How to Use</div> <div> <ul style="list-style-type: none">-Cleaning- <p>To achieve consistent results in coloring, the restoration must be free of dirt and grease and must be completely dry to obtain uniform coloring results.</p> <ul style="list-style-type: none">-Coloring- <p>Choose an appropriate liquid according to the color of the teeth. Shake the liquid bottle well before use. Close the bottle immediately after use to prevent the concentration change.</p> <ul style="list-style-type: none">-Brushing method- <p>For unsintered zirconium oxide restorations, brush the solution with the recommended brush. To get a uniform color, it is better to use a smaller brush. The number of brushes applied determines the amount of shade generated, allowing you to reproduce natural teeth vividly.</p></div>	
<div><div></div>-Dipping method-</div> <div>Aliquot the color solution into the dipping container The restoration should be completely submerged. A dipping time of 20-30 seconds is recommended. Take out the dipped restoration and quickly remove the solution from the surface of the restoration by using a paper towel.</div>	
<div><div></div>-Drying-</div> <div>Colored restorations should be dried before sintering. Observe the following guidelines when handling color solutions. The restoration must be sufficiently dry before sintering. Otherwise, the restoration and/or the furnace may be damaged. Drying using an infrared lamp or drying cabinet. Drying time depends on the temperature and size of the object. Lower temperatures mean longer drying times. Large objects slow down the drying process. If the drying temperature exceeds 140 °C, defects may occur. Recommended drying times are shown in the following table.</div>	

<div><div></div>Drying time depending on the size and temperature of the restoration</div>		
Drying times	Temperature(~70°C)	Temperature(~140°C)
Single unit restorations	≥ 15 min.	5-10 min.
Up to 3-unit bridges	≥ 40 min.	≥ 25 min.

<div><div></div>-sintering-</div> <div>Place the restoration margins upward on the sinter tray, making sure that the restorations do not touch the sides or each other on the sinter tray. Operate the furnace to proceed sintering.</div>	
--	--

<div><div></div>Contraindications</div> <div>All applications not listed above are contraindicated.</div>	
<div><div></div>Packing</div> <div>15ml, 20ml, 30ml, 40ml, 100ml</div>	

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